



Handpiece Holders

Auto IC Molded Handpiece Holder 99-0583-00 Auto and 99-0584-00 Assistant's

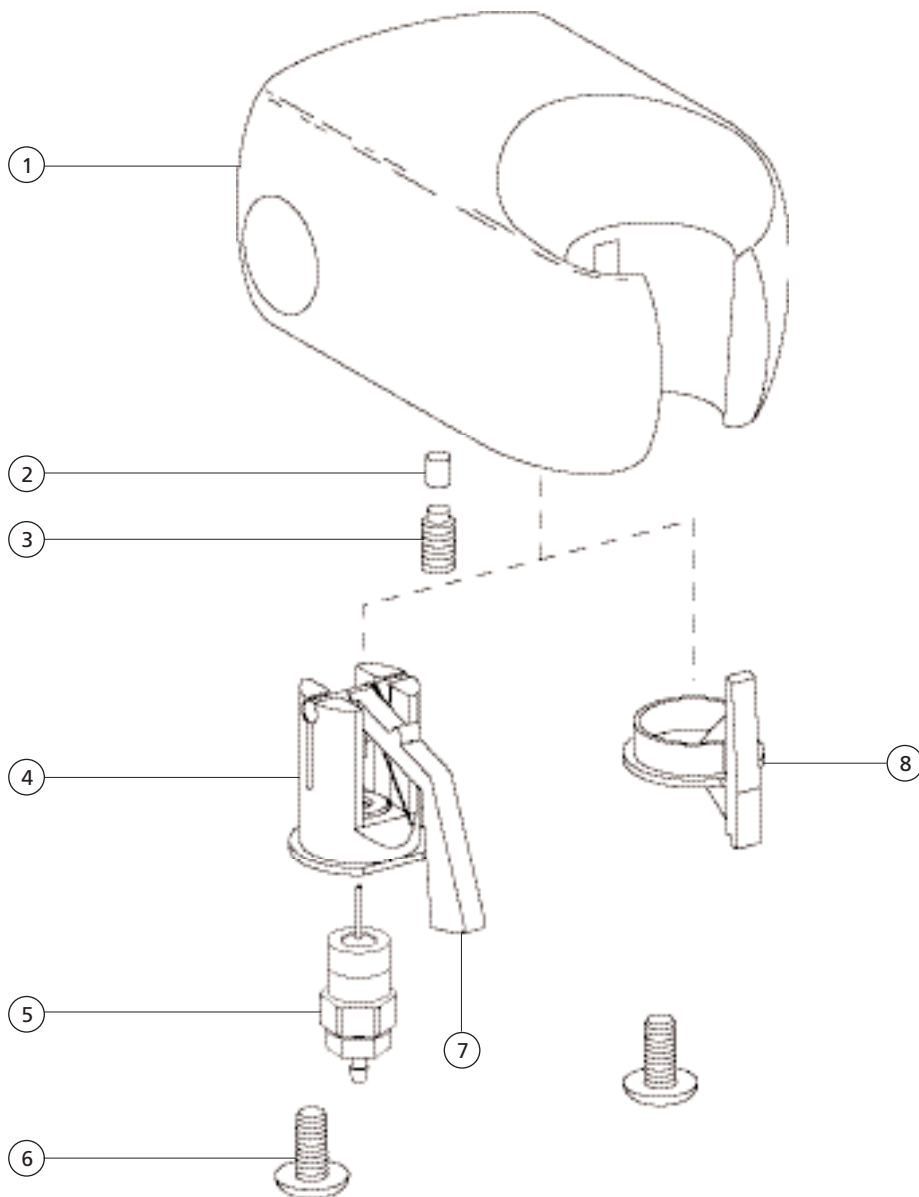


REF. NO.	DESCRIPTION	PART NO.
1	Handpiece Holder, IC, Molded	99-0589-00
2	Friction Pad, Blk, Holder	45-0403-00
3	Setscrew, Soc Cup Pt, 1/4-20 x 3/8	007-042-00
4	Actuator Valve Assy, Auto Holder	99-0592-00
5	Valve Assy, Air Bleed	33-0025-00
6	Screw, Pan Hd Phil 6-32 x 3/8, SST	002-015-00
7	Lever, Auto Holder	99-0586-00
8	Slot Plug, Asst. Holder	99-0587-00

IC Handpiece Holder
99-0572-00 Obsolete (replaced by 99-0583-00)

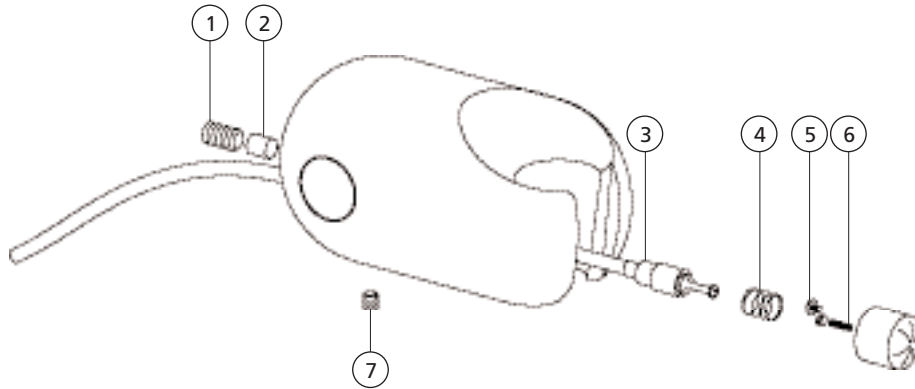


REF. NO.	DESCRIPTION	PART NO.
1	Setscrew, Soc Cup Pt, 1/4 -20 x 3/8	007-042-00
2	Friction Pad, Blk, Holder	45-0403-00
3	Uni-Clamp, 1/8 Poly	025-007-01
4	Spring, Compression, .240 OD x .307 LG	013-066-00
5	Screw, Tr Hd Slot, 2-56 x 1/8 SST	001-002-01
6	Spring, Compression, .088 OD x .250	013-003-01
7	Setscrew, Soc Cup Pt, 6-32 x 3/16	007-002-00

IC Handpiece Holder
99-0582-00 Obsolete (replaced by 99-0584-00)

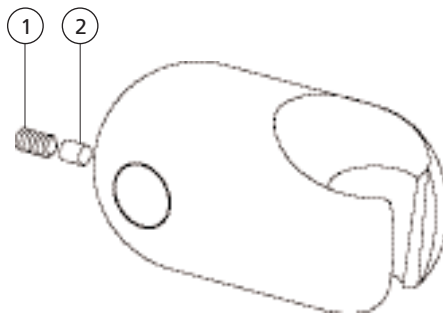


REF. NO.	DESCRIPTION	PART NO.
1	Setscrew, Soc Cup Pt, 1/4-20 x 3/8	007-042-00
2	Friction Pad, Blk, Holder	45-0403-00

Excellence Handpiece Holder 99-0024-XX

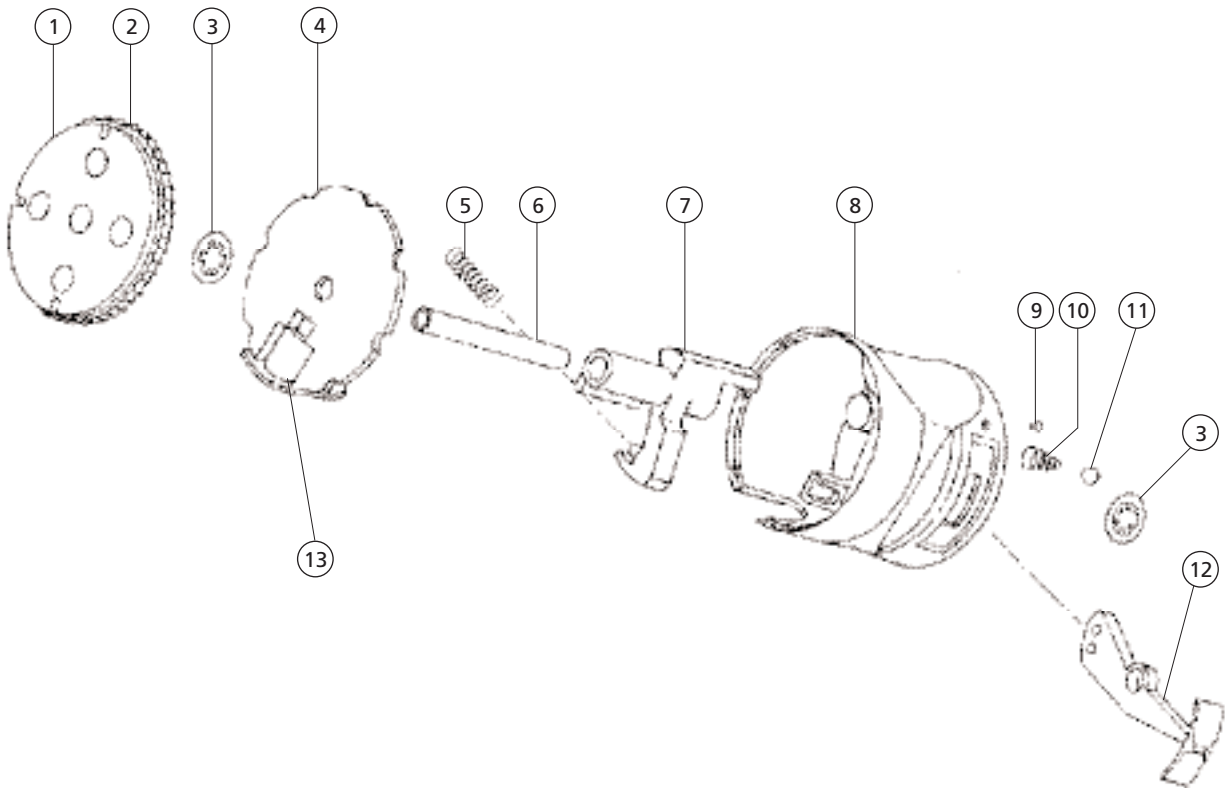


REF. NO.	DESCRIPTION	PART NO.
1	Plate, Retaining, Indexing Wheel	39-0684-00
2	Indexing Wheel, Radial Holder	39-0685-00
3	Retaining Ring, Circ, Ext, 1/4 Dia	010-037-00
4	End Cover, Radial Holder Black	99-0028-00
5	Spring, Compression .180 x .75 LG	013-049-00
6	Shaft, 1.82 Long x .250 OD	99-0029-00
7	Actuator, Radial Holder Black	99-0026-00
	Newport Gray	99-0026-01
8	Body, Radial Holder Black	99-0025-00
	Newport Gray	99-0025-01
9	Plug, Red Color Code, POM	40-0690-00
10	Spr, Conc, Comp, .150 OD x .240 FL	013-013-00
11	Ball, .156 Dia, Stainless Steel	015-006-00
12	Lever, Lockout, Rad Holder Black	99-0027-00
	Newport Gray	99-0027-01
13	Valve Assy, Air Bleed	33-0025-00

Auto and Assistant's Holder 99-0537-XX

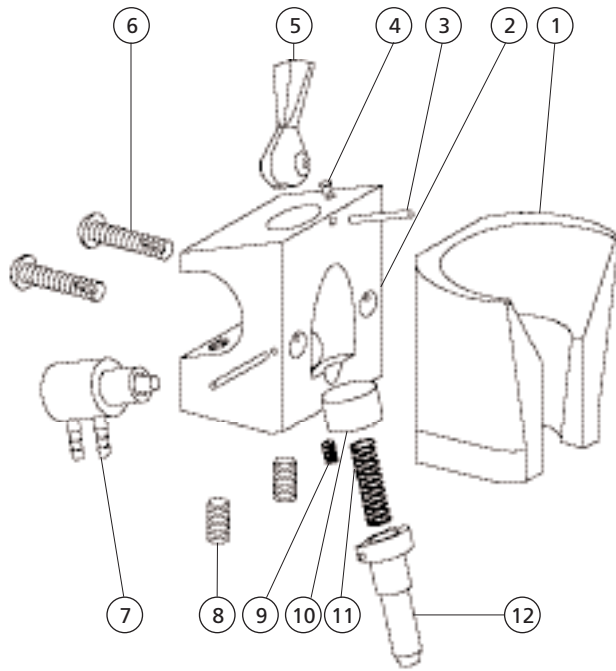


REF. NO.	DESCRIPTION	PART NO.
1	Holder, Handpiece, Auto Black Newport Gray	99-0541-00 99-0541-01
2	Clamp, Auto Holder Black Gray	40-0710-00 40-0710-01
3	Pin, Special, .062 Dia x .530 Lg, Stl	011-041-00
4	Indicator, Plastic, Red	40-0690-00
5	Lever, Plastic, Toggle Valve Black Newport Gray	22-0462-00 22-0462-01
6	Screw Pan Hd Phil #4 x 1/2 Stl, Zinc	003-088-00
7	Micro Valve, 3-Way	33-0138-00
8	Setscrew Soc Cup Pt 10-32 x 1/2 Zinc	007-051-00
9	Setscrew, Soc Cup Pt, 6-32 x 3/16	007-002-00
10	Spacer, .367 OD x .200 ID Black Newport Gray	40-0665-00 40-0665-01
11	Spring, Helical Comp, .18 OD x .750	40-0182-00
12	Actuator, Auto Holder Black Newport Gray	40-0667-00 40-0667-01

Assistant's Holder 99-0528-XX

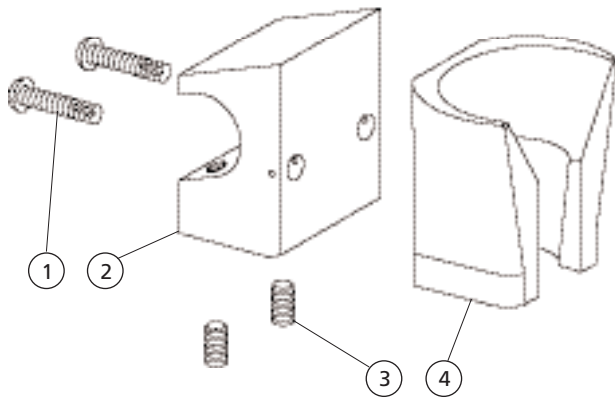


REF. NO.	DESCRIPTION	PART NO.
1	Screw Pan Hd Phil #4 x 1/2 Stl, Zinc	003-088-00
2	Clamp, Holder, Assistant Black Gray	99-0540-00 99-0540-01
3	Setscrew Soc Cup Pt 10-32 x 1/2 Zinc (Used on 99-0528-01)	007-051-00
4	Holder, Handpiece, Asst. Black Gray	99-0539-00 99-0539-01

Tru-Torq Auto Holder Assembly 99-0544-00

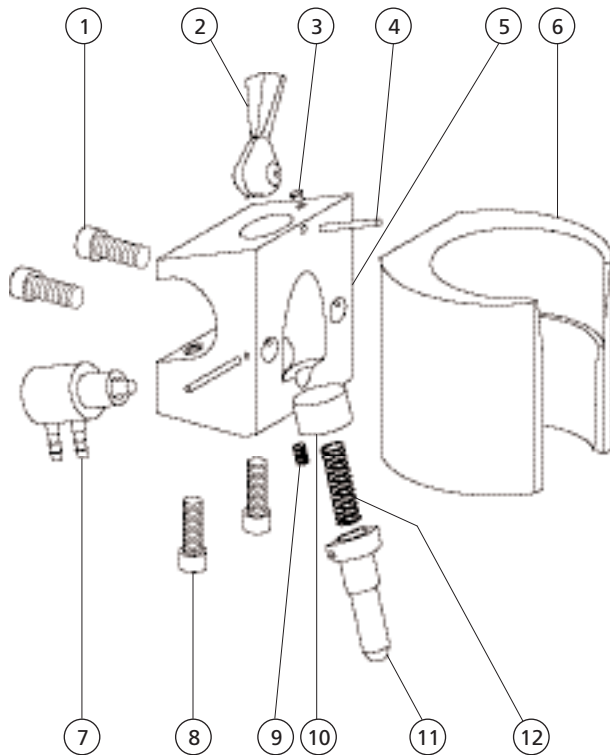

REF. NO.	DESCRIPTION	PART NO.
1	Screw, Soc Hd, 6-32 x 3/8 SST	001-033-00
2	Lvr, Plastic, Toggle, Vlv, Blk	22-0462-00
-	Lvr, Plast, Tgl Vlv, New/Gray, Pur	22-0462-01
-	Lever, Plast, Tgl Vlv, Surf 4, Pur	22-0462-02
3	Indicator, Plastic, Red	40-0690-00
4	Pin, Special, .062 Dia x .530 Lg, Stl	011-041-00
5	Clamp, Auto Holder	40-0710-00
6	Holder, Auto, Black, Tru-Torq	99-0543-00
7	Valve Assembly, Micro, 3-way	33-0138-00
8	Screw, Soc Hd, 6-32 x 9/16 Black	005-105-00
9	Setscrew, Soc Cup Pt, 6-32 x 3/16	007-002-00
10	Spacer, .367 OD x .20 ID, Blk, Pom	40-0665-00
-	Spr, .36 OD x .21D, New/Gra, Pom	40-0665-01
11	Actuator, Auto Holder, Blk, Pom	40-0667-00
-	Actuator, Auto Hlder, New/Gray, Pom	40-0667-01
12	Spring, Helical Comp, .18 OD x .750	40-0182-00

Tru-Torq Manual Holder Assembly 99-0360-00 and 99-0361-00

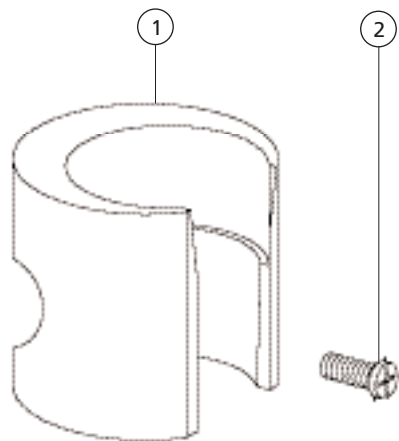

REF. NO.	DESCRIPTION	PART NO.
1	Holder, Tru-Torq White Black	99-0360-00 99-0361-00
2	Screw, Pan Hd Sems Phil, 6-32 x 3/8	001-097-00