PRODUCT NAME: ICX®

A-dec® ICX® tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

KEEP OUT OF REACH OF CHILDREN

WARNING

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals - Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

DIRECTIONS FOR USE:

1. Empty any remaining water left in the bottle.
2. Drop tablet into an empty dental unit water bottle (0.7L tablet in 0.7 liter bottle, 2 L tablet in 2 liter). Avoid touching the tablet with unprotected skin.
3. Fill bottle with water and connect to the dental unit.
4. Wait 60 seconds for the tablet to fully dissolve.

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing.

ICX® is intended for use in general dental applications by providing water to the dental syringe, various handpiece ancillary devices, and cuspidor cupfill. When used according to these instructions for every bottle refill, optimum efficacy for control of odor and foul taste bacteria can be achieved.

FIRST AID:

If solid product gets in eyes, hold eye open and rinse slowly and gently with water 15 to 20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage disposal. Dispose of empty outer foil pouch in the trash. Store at room temperature out of direct sunlight; **DO NOT** store above 30°C (86°F).

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