



# A-dec® Flush Dock

## Preparing for your next patient just got easier.

Sometimes a simple solution is the best solution. Take A-dec Flush Dock for example. With specified slots to hold handpieces and A-dec syringes, you can flush, purge, and consistently follow recommended waterline protocol, while leaving your hands free for other important tasks. It's that easy.

### When every minute counts, count on A-dec Flush Dock.

**HANDS FREE** — Paired with the A-dec 500 Pro or 300 Pro timed flush feature, the Flush Dock provides the staff with a hands-free solution that doesn't require a sink, HVE or cuspidor. It's portable, stand-alone, and easy to move between rooms. At the end of the day, simply rinse and repeat.

**AUTOMATIC** — When a syringe is inserted into the holder, a built-in feature depresses the water button automatically triggering the syringe flush. Handpiece lines may be started by holding the flush toggle or via the timed flush feature on the DS7 and CP5i (Pro) control pads, which allow customized, pre-set flush timers to start the process. With the Flush Dock in place, it's ready to go.

**CONSISTENT** — Easy to use, efficient and consistent, the Flush Dock helps reinforce waterline maintenance health and safety and can also be used for dental unit waterline purging after A-dec ICX Renew® waterline shock treatments.

#### Key Features:

- Syringe pocket is specifically designed to press and hold the A-dec syringe water button (A-dec syringes only).
- Handpiece holders can accommodate a wide spectrum of handpiece tubing nuts and ends.
- Container top is removeable to easily clean the holder slots.
- Lid has space for two traditional or one Continental® syringe, allowing doctor and assistant delivery flush at the same time, along with four handpiece slots (remove all handpieces, motors, and couplers to flush).



Contact your A-dec representative for more information.