**A-dec ICX Restore Solution 1 and Solution 2** is a specially formulated dental unit waterline shock treatment. It is intended to remove build-up of contamination from the waterlines. Easy to use and no diluting required. Always use ICX Restore Solution 1 and 2 together. Do not add water. This product is to be used in conjunction with regular testing of dental unit water.

**NOTE** Do not use with a waterline maintenance cartridge or straw.

**DIRECTIONS FOR USE. Always use Solution 1 and 2 together.**

### Applying ICX Restore to a Dental Unit's Waterlines

1. Turn off the delivery system master toggle; then remove and empty the water bottle.
2. Remove air/water syringe tips, handpieces, detachable motors and couplers, and ultrasonic instruments from their tubing.
3. Add one bottle (118 mL) of ICX Restore Solution 1 and one bottle (118 mL) of ICX Restore Solution 2 into the empty water bottle.
4. Cover the opening of the water bottle and carefully swirl the solution, coating all surfaces of the water bottle.
5. Reattach the water bottle to the unit and turn on the master toggle.
6. Run ICX Restore through the system until the blue solution flows out of the air/water syringe(s), all of the handpiece tubing and the cuspidor's cupfill waterline. Leave remaining solution in the water bottle.
7. Turn off the master toggle and allow the solution to remain in the unit for 12 to 96 hours.

**NOTE** Place a cloth or paper towel underneath the handpiece tubing to catch potential drips.

8. After 12 to 96 hours, turn on the master toggle and flush remaining ICX Restore solution through all air/water syringe(s), all handpiece tubing, and the cupfill waterline.
9. Turn off the master toggle, remove the water bottle, discard any remaining solution, and thoroughly rinse the bottle with water.
10. Refill the bottle with water, reattach it to the unit; then turn on the master toggle.
11. Completely flush all air/water syringe(s) and all handpiece tubing into a sink or cuspidor, and run the cupfill until the water is clear and the bottle is empty.
12. Once treatment with ICX Restore is completed, add an ICX® tablet to the water bottle and refill it with water.

**IMPORTANT.** For **new equipment**, apply one treatment before initial use. For **existing equipment**, apply three treatments when your water quality test results exceed your water quality action level. You may use the dental unit between treatments. For proactive preventive maintenance, you may apply one treatment on a routine basis.

### SOLUTION 2

**WARNING** Causes eye irritation.

### PRECAUTIONS

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice.

## **STORAGE AND DISPOSAL. Solutions 1 and 2:**

**Product Storage.** Keep the product in the closed original container when not in use. Store in a cool, dry area away from potential sources of heat, open flames, sunlight or other chemicals. Store between 5 °C (41 °F) and 30 °C (86 °F).

**Product Disposal.** Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.

**Container Handling.** Nonrefillable container. Do not reuse or refill this container. Offer container for recycling if available or puncture and dispose of in a sanitary landfill, or by incineration.

**Regulatory Information and Warranty:** For required regulatory Information and the *A-dec warranty*, see the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00) available in the Resource Center at [www.a-dec.com](http://www.a-dec.com).

A-dec, A-dec 360, the A-dec logo, and ICX Restore are trademarks or registered trademarks of A-dec, Inc. in the U.S. and other countries.

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All rights reserved
Copyright 2021 A-dec, Inc.