# Owner's Guide

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Authorized Service Center:

A-dec, Inc  
Newberg, Oregon 97132 USA  
2601 Crestview Drive  
t 1-800-547-1883  
1-503-538-7478  
f 1-503-538-0276

Manufacturer:

W&H Sterilization S.r.l  
Italy, I-24060 Brusaporto(BG)  
Via Bolgara, 2  
t +39/035/66 63 000  
f +39/035/50 96 988  
e office@wnhster.it  
wh.com

Importer:

A-dec, Inc  
Newberg, Oregon 97132 USA  
2601 Crestview Drive  
t 1-800-547-1883  
1-503-538-7478  
f 1-503-538-0276

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*Lisa* MB17/22

See information specific to MB22  
on pages 21, 36, 43, and 44.

# Owner's Guide

## *Lisa* MB 17

### WATER STEAM STERILIZER

**©2007 BY A-DEC INC. ALL RIGHTS RESERVED.**

2601 Crestview Drive, Newberg, OR 97132. Printed in the USA.

A-dec makes no warranty of any kind with regard to the content of this document, including but not limited to, the implied warranties of merchantability and fitness for a particular purpose. A-dec Inc. shall not be held liable for any errors contained herein or any consequential or other damages concerning the furnishing, performance or use of this material. The information in this document is subject to change without notice. If you find any problems with this document, please report them to us in writing. A-dec Inc. does not warrant that this document is error-free.

All other non-A-dec products or services mentioned in this document are covered by the trademarks, service marks, or product names designated by the companies marketing those products.

A word about this Owner's Guide: Every effort has been made to make this document as comprehensive and factual as possible.

We reserve the right, however, to make changes at any time without notice in prices, colors, materials, equipment specifications, models and availability. Since some information may have been updated since the time of printing, please check with A-dec for complete details.

**SUMMARY**

<b>1.</b>	<b>UNPACKING.....</b>	<b>5</b>
1.1.	UNPACKING THE STERILIZER .....	5
1.2.	UNPACKING THE ACCESSORIES .....	6
<b>2.</b>	<b>SETUP .....</b>	<b>7</b>
2.1.	INSTALLING THE STERILIZER.....	7
2.2.	CONNECTING THE ELECTRICAL SUPPLY .....	8
2.3.	CONNECTING THE LISA LOG64 (optional).....	9
<b>3.</b>	<b>DESCRIPTION.....</b>	<b>10</b>
3.1.	RECOGNIZING THE FRONT OF THE STERILIZER .....	10
3.2.	RECOGNIZING THE BACK OF THE STERILIZER .....	11
<b>4.</b>	<b>USE .....</b>	<b>12</b>
4.1.	LOCATING THE MAIN MENU.....	12
4.2.	USING THE WATER TANKS .....	13
4.3.	FILLING THE MAIN WATER TANK .....	14
4.4.	DRAINING THE USED WATER TANK .....	15
4.5.	PROGRAMMING .....	16
4.5.1.	<b>SELECTING A LANGUAGE .....</b>	<b>17</b>
4.5.2.	<b>PROGRAMMING THE NAME OF THE DOCTOR / CLINIC .....</b>	<b>17</b>
4.5.3.	<b>PROGRAMMING THE DATE AND TIME .....</b>	<b>18</b>
4.5.4.	<b>PROGRAMMING THE PRINTER .....</b>	<b>18</b>
4.5.5.	<b>ADJUSTING THE BRIGHTNESS OF THE TOUCH-SCREEN .....</b>	<b>19</b>
4.5.6.	<b>PROGRAMMING THE STAND-BY MODE .....</b>	<b>19</b>
4.5.7.	<b>USING THE MAINTENANCE PROGRAM SCREEN .....</b>	<b>20</b>
4.5.8.	<b>RUNNING SERVICE PROGRAMS .....</b>	<b>20</b>
4.6.	CYCLE SELECTION .....	21
4.6.1.	<b>CYCLE TABLE .....</b>	<b>21</b>
4.6.2.	<b>STARTING A CYCLE .....</b>	<b>22</b>
4.6.3.	<b>UNDERSTANDING THE CYCLE PROCEDURE .....</b>	<b>23</b>
4.6.4.	<b>COPIES OF CYCLE PRINTOUTS .....</b>	<b>23</b>
4.7.	MANUAL STOP .....	24
4.8.	LOCATING CYCLE DATA INFORMATION .....	25
4.9.	LOCATING CYCLE DATA SUMMARY (end of the cycle) .....	26
<b>5.</b>	<b>MESSAGES .....</b>	<b>27</b>
<b>6.</b>	<b>ALARMS .....</b>	<b>28</b>
6.1.	DESCRIPTION OF THE ALARMS .....	29
<b>7.</b>	<b>USER MAINTENANCE.....</b>	<b>31</b>
7.1.	PERFORMING MAINTENANCE .....	31
7.2.	CLEANING THE DOOR SEAL .....	32
7.3.	CLEANING THE CHAMBER, TRAYS, AND TRAY HOLDER.....	32
7.4.	CLEANING THE EXTERNAL PARTS .....	32
7.5.	REPLACING THE FILTER .....	32
7.6.	REPLACING THE DOOR SEAL.....	33
<b>8.</b>	<b>SERVICING CONDUCTED BY THE APPROVED TECHNICIAN .....</b>	<b>34</b>
<b>9.</b>	<b>BREAK-DOWN GUIDE .....</b>	<b>35</b>
<b>Appendix 1</b>	<b>TECHNICAL CHARACTERISTICS.....</b>	<b>36</b>
<b>Appendix 2</b>	<b>PREPARATION OF THE LOAD .....</b>	<b>37</b>
<b>Appendix 3</b>	<b>STERILIZATION MONITORING .....</b>	<b>38</b>
<b>Appendix 4</b>	<b>VACUUM TEST .....</b>	<b>39</b>
<b>Appendix 5</b>	<b>WATER QUALITY .....</b>	<b>40</b>
<b>Appendix 6</b>	<b>ACCESSORIES.....</b>	<b>41</b>
<b>Appendix 7</b>	<b>SYMBOLS .....</b>	<b>42</b>
<b>Appendix 8</b>	<b>WARRANTY .....</b>	<b>43</b>
<b>Appendix 9</b>	<b>A-dec W&amp;H Lisa Installation Checklist.....</b>	<b>44</b>

English

---

## RECOMMENDATIONS – SAFETY

The purpose of this Owner's Guide is to provide you with the information you need to ensure:

- Proper installation and set-up.
- Optimal use.
- Safe and reliable operation.
- Regular and correct servicing requirements.

## INTENDED USE

The sterilizer is intended to be used for steam sterilization of solids, textiles and hollow items unwrapped, single wrapped or double wrapped.

## GENERAL RECOMMENDATIONS – SAFETY



- The user is responsible for operating and servicing the sterilizer in accordance with the instructions listed in this manual.
- The sterilizer has not been designed for the sterilization of liquids.
- The sterilizer has not been designed to operate in the presence of gas or explosive vapors.
- The chamber is automatically heated to 170°F as soon as the sterilizer is switched ON.
- The trays and the load will still be hot at the end of each cycle. Use the tray holder to remove each tray from the sterilizer chamber.
- Comply with chamber loading and cycle selection (see *Cycle selection*).
- Do not remove the instruction plate or any label from the sterilizer.
- Do not pour water or any other liquid over the sterilizer.
- Unplug the electrical cord before inspecting or servicing the machine.
- Repairs and maintenance must be performed by an authorized A-dec/W&H Dealer using only original spare parts.
- In case of transport, drain both water tanks completely, allow the sterilization chamber to cool down and preferably use the original packaging (see *Filling the main water tank* and *Draining the used water tank*).

## 1. UNPACKING

The sterilizer, the accessories, the Owner's Guide and the warranty card are supplied in a sturdy box.

Check the condition of the packaging on receipt. Contact the carrier immediately and inform your supplier if the outer packaging is damaged.


### 1.1. UNPACKING THE STERILIZER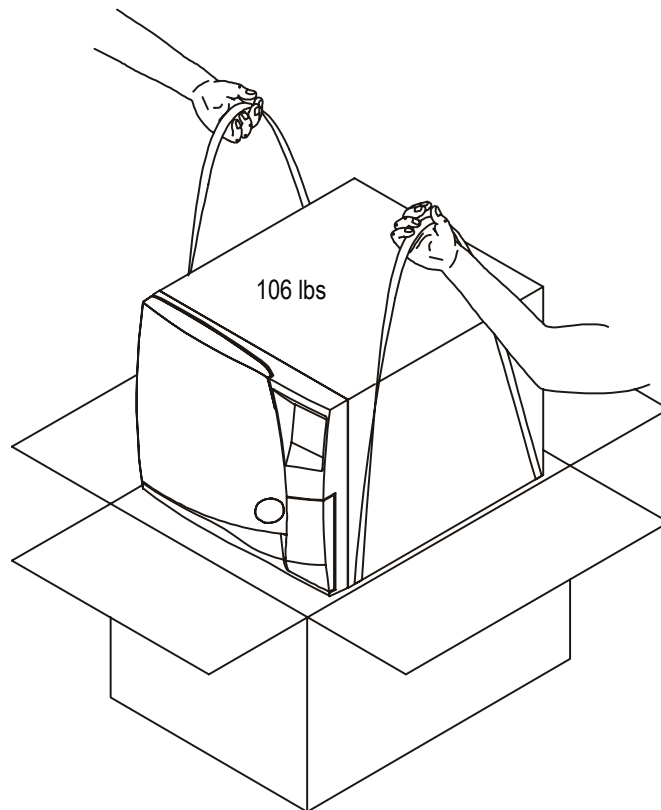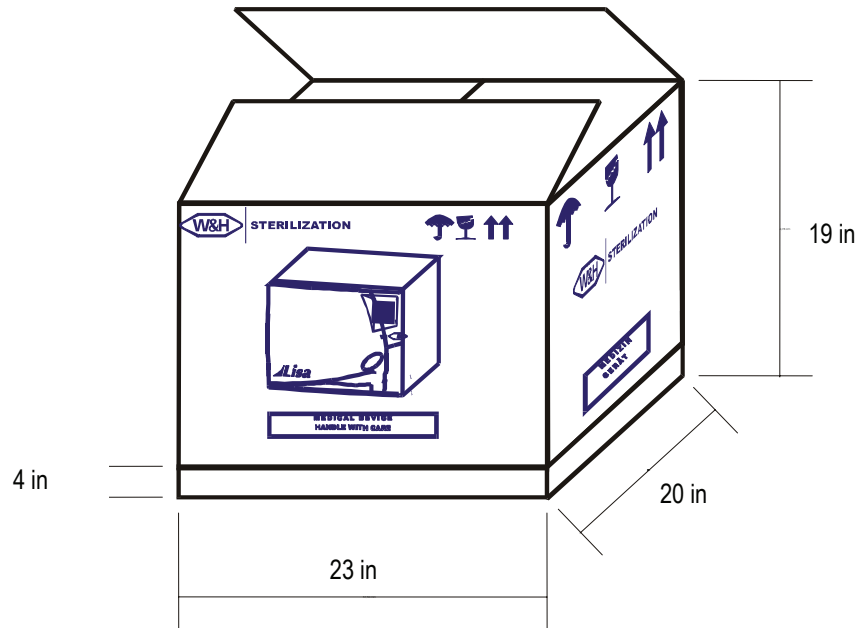

## 1.2. UNPACKING THE ACCESSORIES

1. Open the door and remove the accessories from the sterilization chamber.
2. Check the contents:

- ① Five anodized aluminum trays
- ② Reversible rack for 3 cassettes or 5 trays
- ③ Tray holder
- ④ Drain tube
- ⑤ Power cord
- ⑥ Funnel
- ⑦ Owner's Guide and warranty card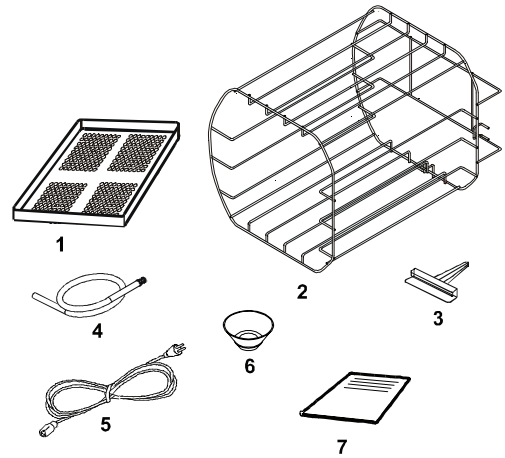



3. Insert the rack into the sterilization chamber and click it into position.

The rack is reversible and can be positioned to take either:

5 trays horizontally or 3 cassettes vertically

or

3 trays / cassettes horizontally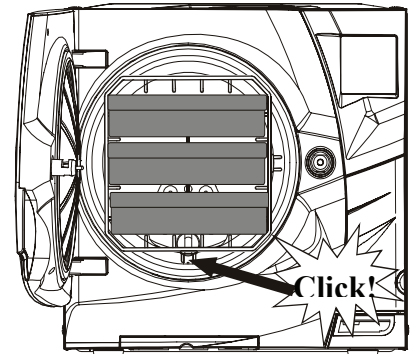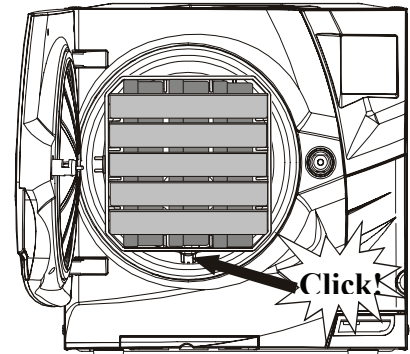


### Usable space

This equates to the usable volume of the chamber accommodating the sterilizable load.

This volume is equivalent to a pipe with the following dimensions:

L: 7.5 in, H: 8 in, W: 12 in (i.e., 13 quarts).

This volume is identical for all cycles and all types of load.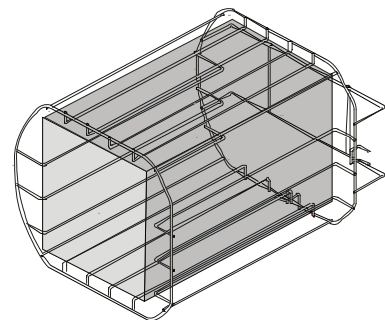


## 2. SETUP

### 2.1. INSTALLING THE STERILIZER

The sterilizer has been calibrated and tested in the factory. It does not require calibration during installation.

Working temperature range: 50°F to 104°F / Humidity: 0 to 90%  
 Storage temperature range: -4°F to 140°F / Humidity: 0 to 90% (water tanks empty)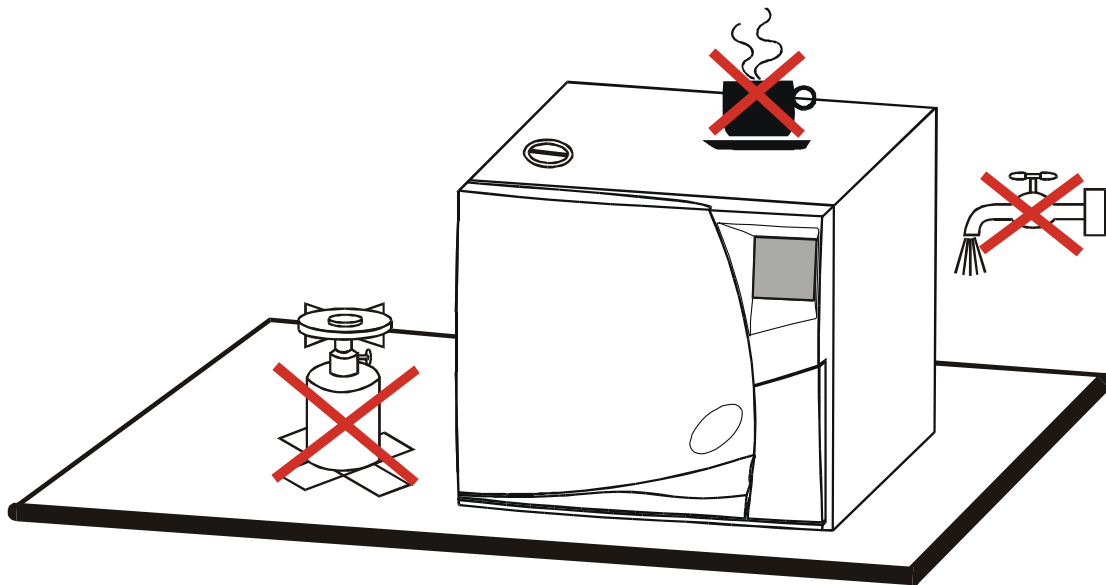

#### Install the sterilizer as outlined below:

1. Install the sterilizer on a flat and level surface. Do not level the sterilizer itself but the surface it is placed on.  
 The maximum weight of the sterilizer with the main water tank full and the chamber fully loaded is 125 lbs.
2. Leave a gap of two inches at the back and one inch on each side of the sterilizer to ensure adequate ventilation.
3. Do not place the sterilizer near a sink or in a location where it is likely to be splashed.
4. Install the sterilizer in a well-ventilated room.
5. Keep away from all sources of heat.


## 2.2. CONNECTING THE ELECTRICAL SUPPLY

To assure proper function of the Lisa MB17, the power supplied must be a consistent source of single phase 230/240 VAC +/-10% (207–264 VAC), 50/60 Hz, minimum 10 Amp. on a dedicated circuit.

Voltages below 230/240 VAC will increase the total cycle time of a selected sterilization cycle. Voltages under 215 VAC are subject to voltage drops caused by line voltage irregularities which may cause sterilizer errors.

To ensure ideal cycle times and to avoid error codes caused by voltage drops when line voltages are under 215 VAC, the installation of a buck boost transformer (to bring voltage to 230/240 VAC) is recommended.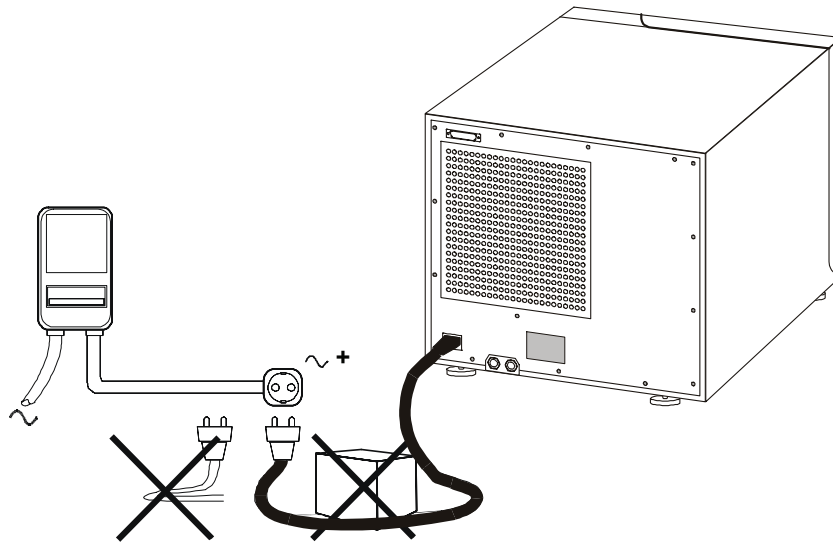


- The electrical installation must comply with the current local/national standards.
- If unclear, have the installation checked by a qualified technician.
- Ensure the power supply is properly grounded.
- The maximum power consumption of the sterilizer is 2100 W (9.2A).
- Only use the power cord delivered with the sterilizer.
- Do not plug other equipment into the same socket.
- Avoid sharp bends or twists in the electrical cable.

## 2.3. CONNECTING THE LISA LOG64 (optional)

The Lisa Log64 is used to digitally save sterilization cycle parameters on a memory card (SD card) for cycle data recording and documentation. The cycle data can be transferred from the memory card to a computer.

Place the recorder on top of the sterilizer or next to it. Do not cover any of the sterilizer tank holes and do not place the device behind the sterilizer to prevent damage from heat.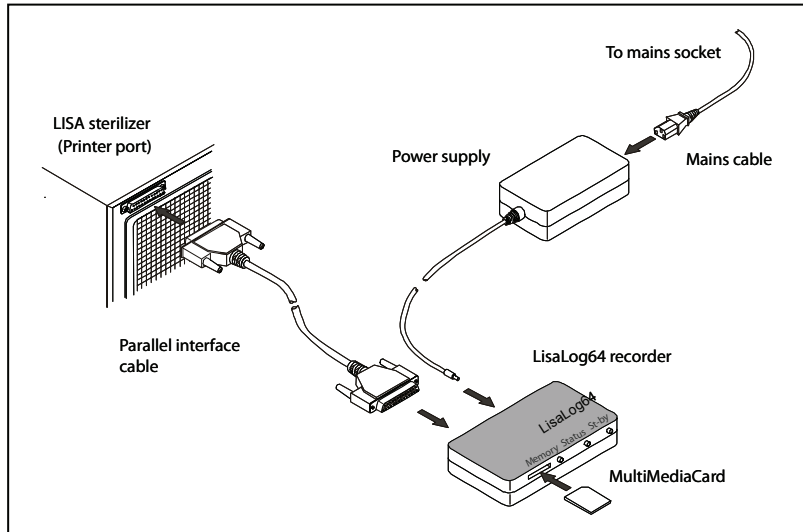

### Connecting the Lisa Log64

1. Turn off the Lisa sterilizer.
2. Attach one end of the parallel interface cable to the parallel port of the sterilizer, located at the rear/top-right corner, and the other end of the interface cable to the parallel port of the recorder module.
3. Connect the power supply to the recorder module.
4. Switch on the sterilizer.
5. Insert the memory card (SD card) into the recorder card slot as shown below. Use memory cards with storage capacity of 128MB or less. Cards with higher storage capacity will not work with the device.


Cycle data will be automatically saved on the memory card. You do not need to change any of the settings on the sterilizer. Transfer cycle data from the memory card to your computer or network periodically. There is no special software required for cycle data management; simply open single data files by double-clicking on them to view cycle data.

The system saves one data file for every cycle that has been run on the sterilizer. The following information can be viewed once a data file is opened:


- Name of doctor / clinic
- Serial number of the sterilizer
- Cycle number
- Date and time at the beginning and at the end of the cycle
- Cycle name
- Duration, temperature and pressure of the various cycle phases
- Comments on cycle outcome

### 3. DESCRIPTION


#### 3.1. RECOGNIZING THE FRONT OF THE STERILIZER

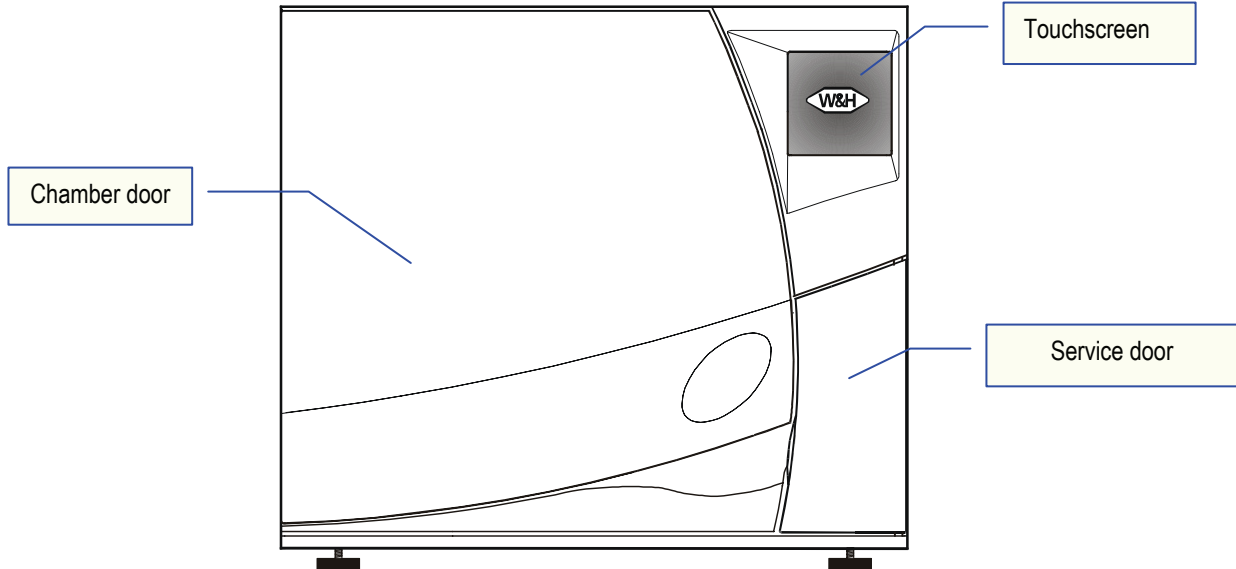#### Inside service door
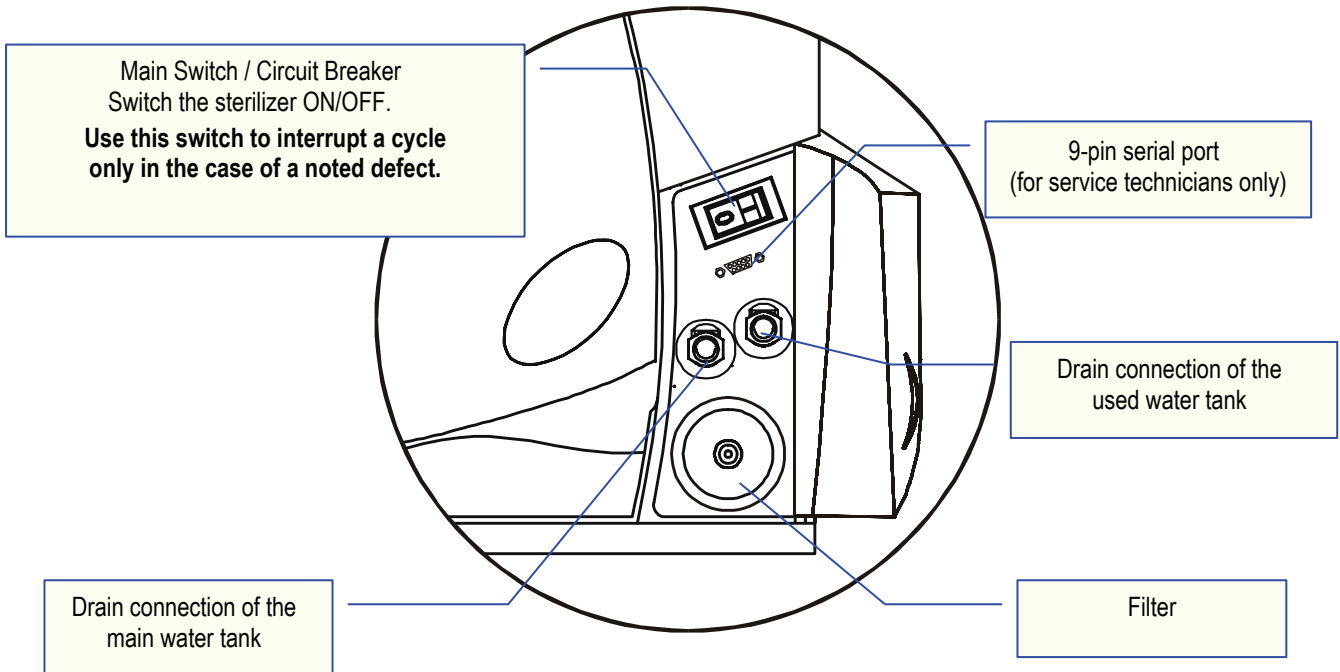
### 3.2. RECOGNIZING THE BACK OF THE STERILIZER

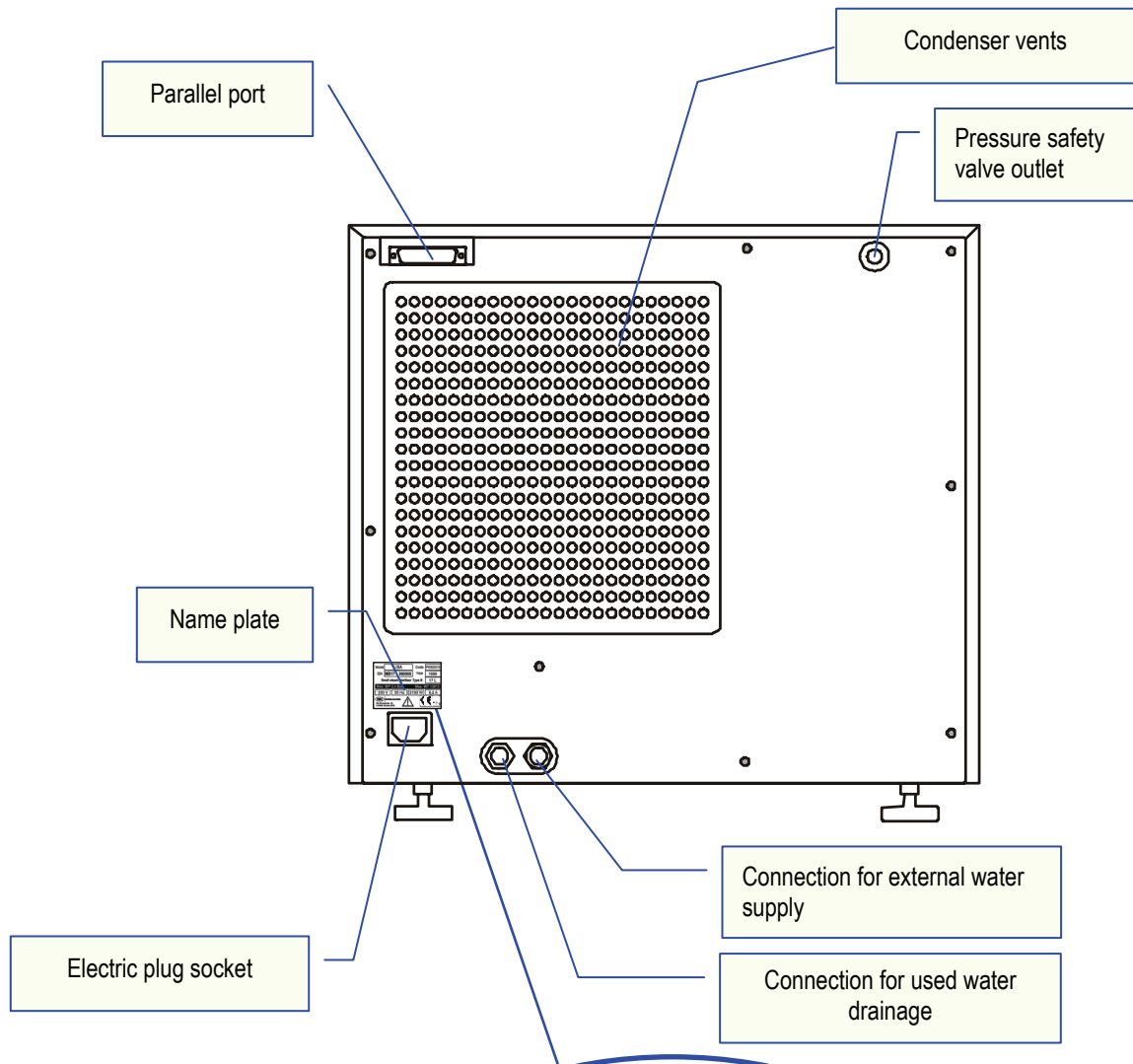English


Model / Modèle	<b>LISA MB 17</b>	Code	<b>P000010</b>
SN	<b>AA-XXXX</b>	Year / Année	<b>AAAA</b>
<b>Small Steam Sterilizer</b>		<b>Type B</b>	<b>17 l / 18 qt</b>
<b>Petit Sterilisateur à Vapeur</b>			
<b>Max. W.P. / P.Max</b>		<b>Max. W.T. / T.Max.</b>	
<b>2.4 bar / 34.8 psi</b>		<b>138 °C / 280.4 °F</b>	
<b>230/240 Vac</b>	<b>50 / 60 Hz</b>	<b>2100 W</b>	
	W&H Sterilization s.r.l. Italy, I - 24060 Brusaporto, Bergamo (BG) Via Bolgara, 2 t +39 035 66 63 000 f +39 035 50 96 988 e office@wnhster.it	 0051	 conseil certifié
			<b>Made in Europe</b> <b>Produit en Europe</b>

## 4. USE

- NOTE:**
- For detailed information on load preparation prior to sterilization, please view Appendix 2 "PREPARATION OF THE LOAD"
  - For detailed information on sterilization monitoring please view Appendix 3 "STERILIZATION MONITORING"

### 4.1. LOCATING THE MAIN MENU

Activate the main switch; the selection screen appears.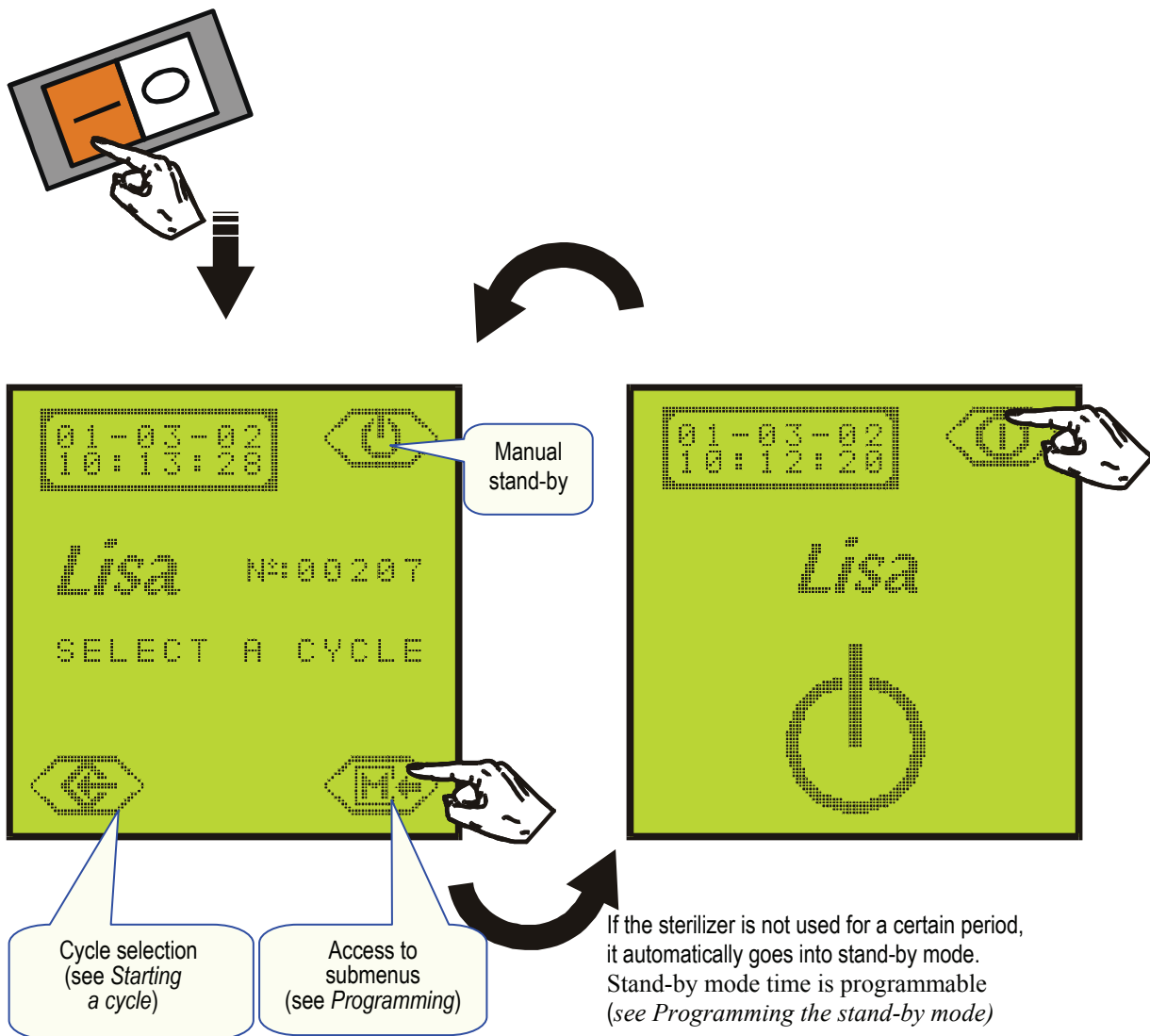

English

- NOTE:** When the sterilizer is switched ON, the chamber automatically heats to 170°F. Pre-heating starts at room temperature and takes approximately 10 minutes. It is possible to select and start a cycle before the end of pre-heating. The sterilizer will continue the pre-heat phase and automatically start the selected cycle after pre-heating is complete.


## 4.2. USING THE WATER TANKS

The sterilizer is equipped with two independent water tanks of equal volume – 4 quarts.

The left tank (main tank) needs to be filled with distilled or demineralized water. It is fitted with two level sensors, minimum (0.5 quarts) and maximum (3.7 quarts).

The right tank (used water tank) contains the used water collected at the end of each cycle. It is fitted with a maximum level sensor (3.7 quarts).

The two tanks connect to drain connections located behind the service door.

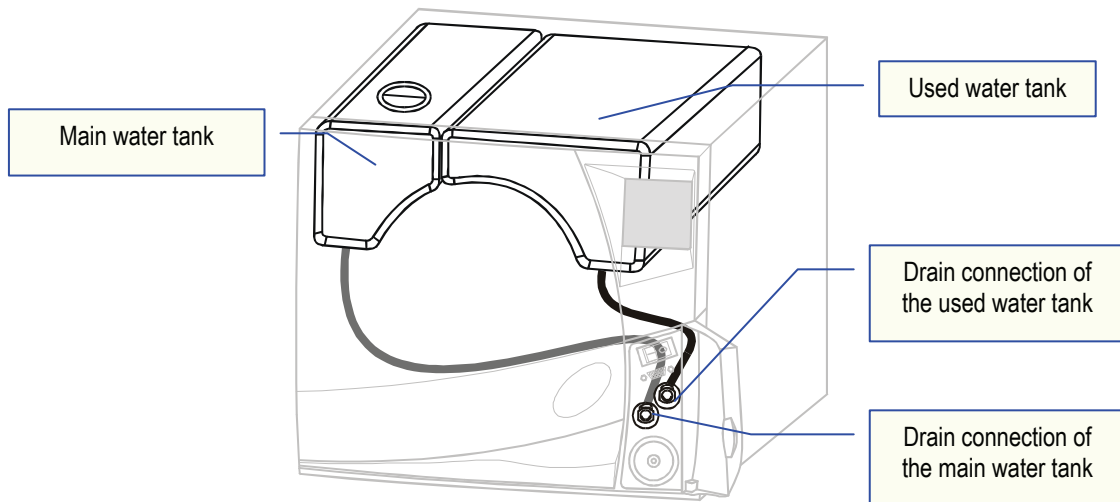The water consumption per cycle varies depending on the type and mass of the sterilized load.

**NOTE:** The sterilizer will run for approximately eight cycles before the main tank needs refilling.

The following message displays on the screen when the minimum water level is reached in the main tank. While the message displays, it is not possible to initiate sterilization cycles.

### 4.3. FILLING THE MAIN WATER TANK

**ATTENTION:** Use distilled or demineralized water only! The mineral content of water used for sterilization needs to be lower than 10 ppm (parts per million) or lower than 15 Microsiemens/cm conductivity (see *Appendix 6*).


1. Remove the cover of the main water tank located on the top of the sterilizer.

2. Attach the funnel and fill the tank with approximately 3.25 quarts of distilled or demineralized water.

Once the tank is full, an audible tone will sound.

3. Stop adding water as soon as you hear the audible tone.

4. Drain the used water tank.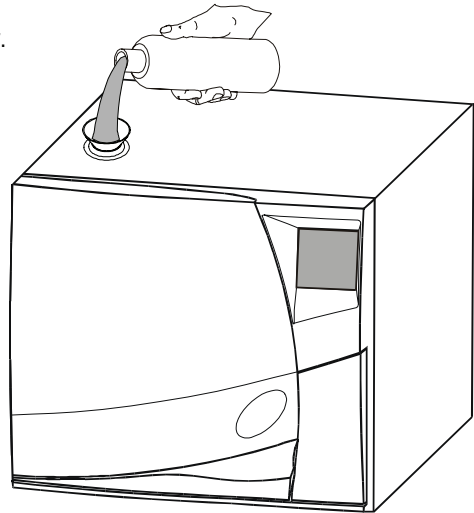

**NOTE:** If the sterilizer is not used for more than three days, both water tanks must be completely drained in order to avoid algae growth or any other deposits.


#### 4.4. DRAINING THE USED WATER TANK

The following message displays on the screen when the maximum water level is reached in the used water tank. While the message displays, it is not possible to initiate sterilization cycles.


1. Open the service door.
2. Insert the drain tube into the quick disconnect coupling drain connection of the used water tank (right).
3. Allow the entire contents of the tank to empty and discard the used water.

**DO NOT RE-USE THE USED WATER IN THE STERILIZER!**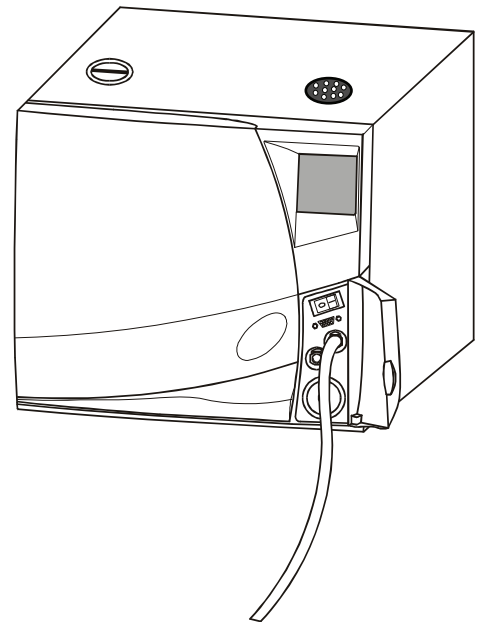

4. Disconnect the drain tube by pressing the push button on the drain connection.


NOTE: If needed, the main water tank can be refilled while the used water tank is draining.

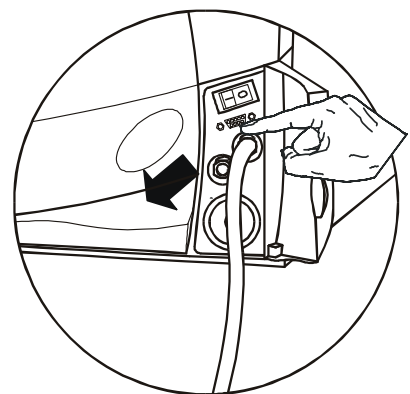
## 4.5. PROGRAMMING

1. On the selection screen, press **M**.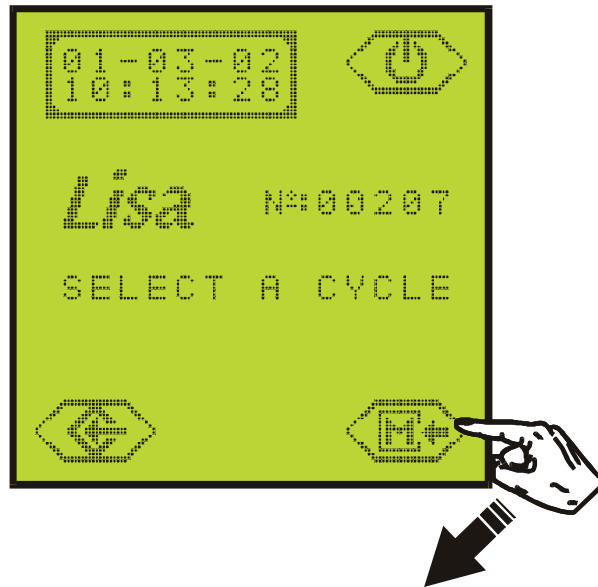


The program menu displays.

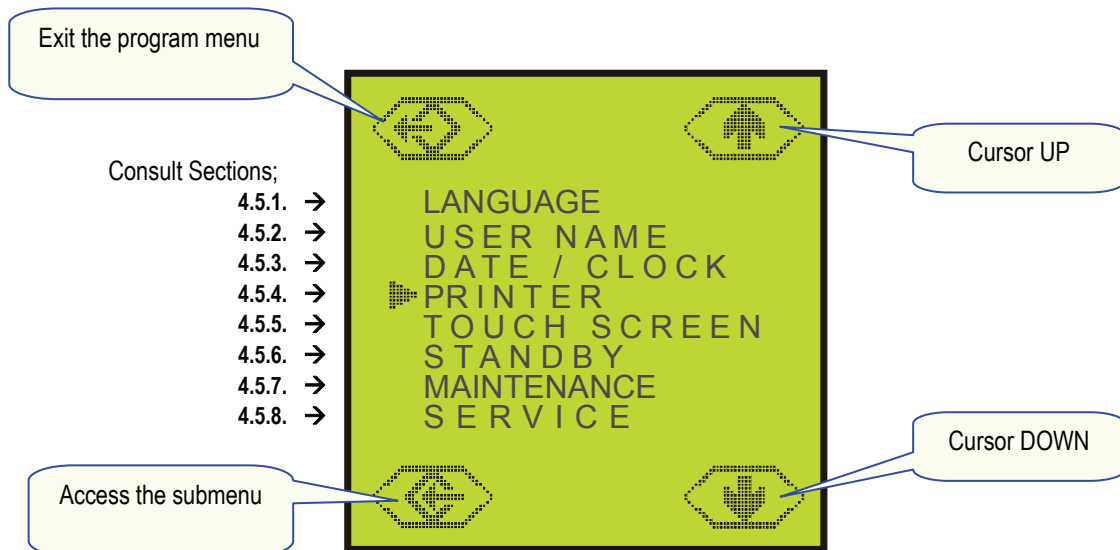
2. Using **UP** and **DOWN**, place the cursor (>) in front of the desired section.  
Confirm the selection by pressing the submenu access key.

3. Refer to the relevant chapter for the selected submenu.

#### 4.5.1. SELECTING A LANGUAGE
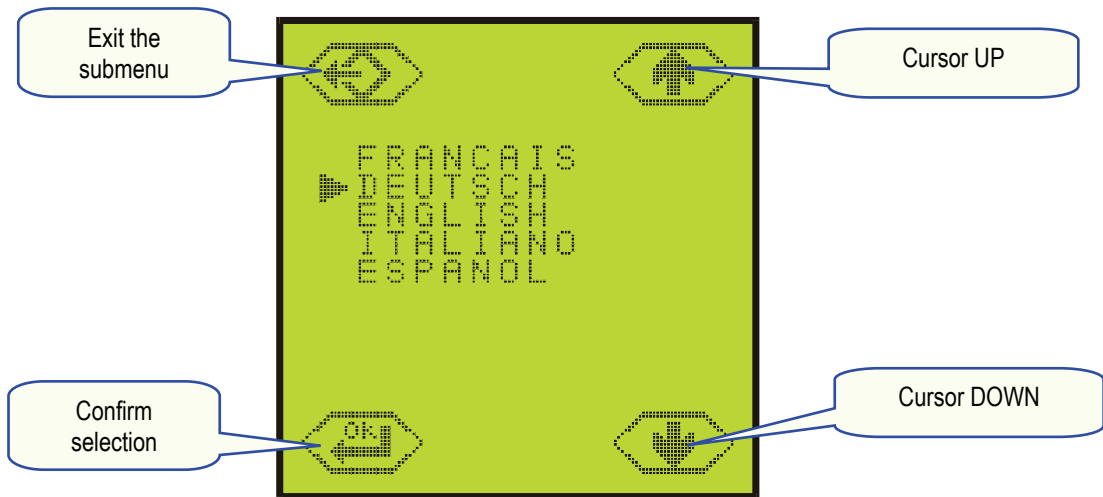
#### 4.5.2. PROGRAMMING THE NAME OF THE DOCTOR / CLINIC

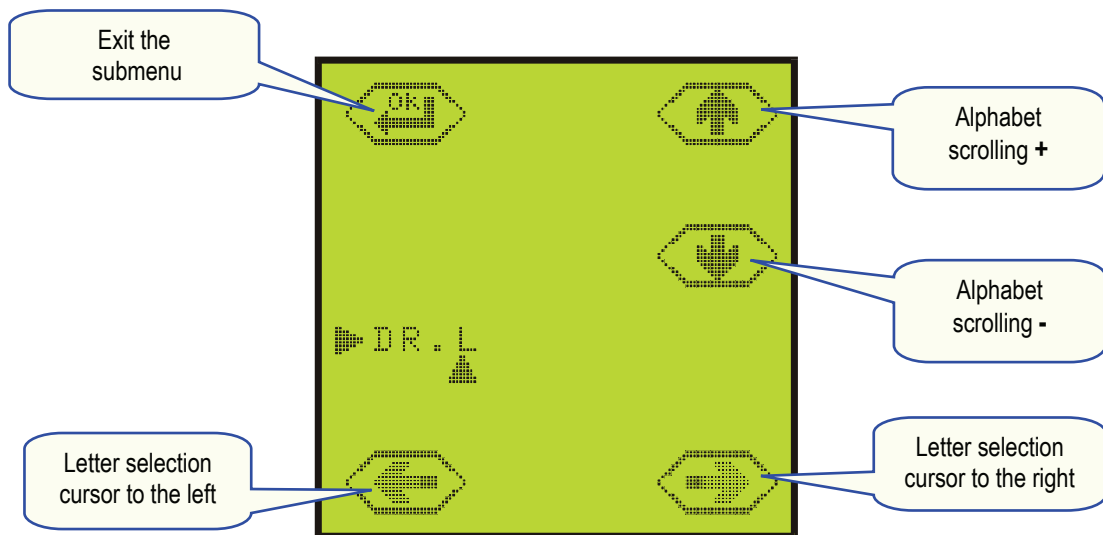
English


### 4.5.3. PROGRAMMING THE DATE AND TIME

It is essential to set the date and time especially if the sterilizer is connected to a data logger. These parameters are systematically printed at the beginning and at the end of each cycle report.


Place the selection cursors under the data that is to be adjusted. Increase or decrease the value, move the cursors under the next value and repeat the operation. Confirm the adjustment by pressing OK.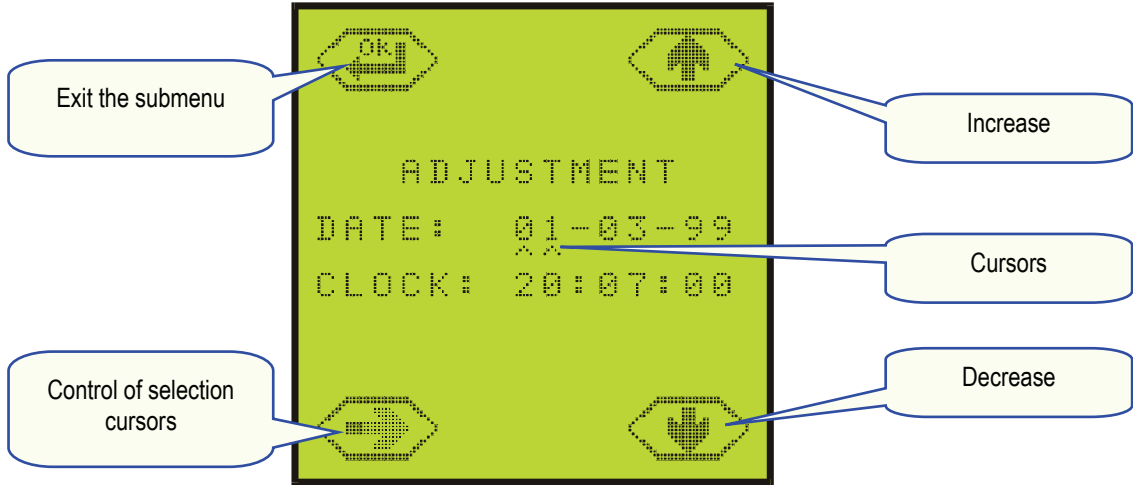

### 4.5.4. PROGRAMMING THE PRINTER

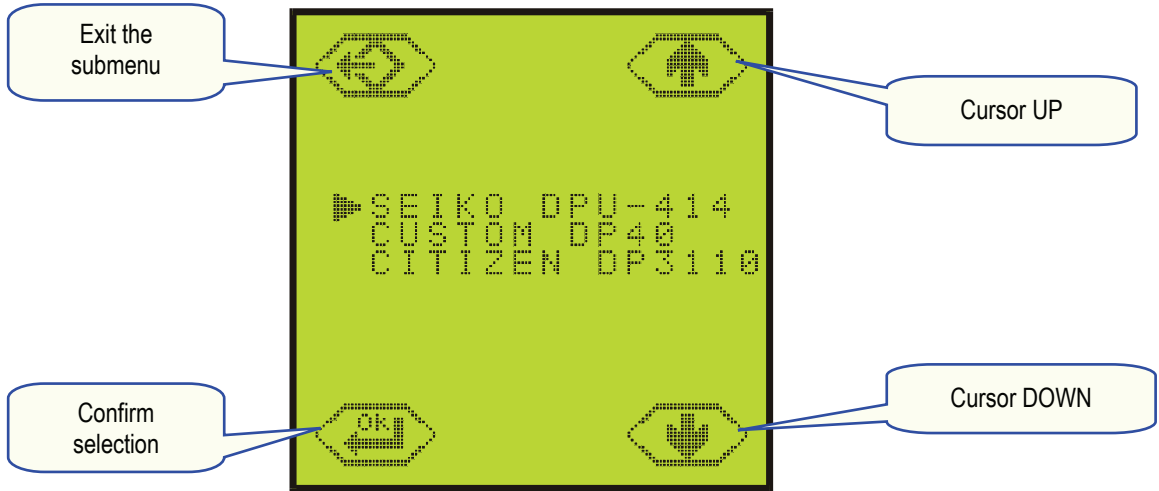
#### 4.5.5. ADJUSTING THE BRIGHTNESS OF THE TOUCH-SCREEN

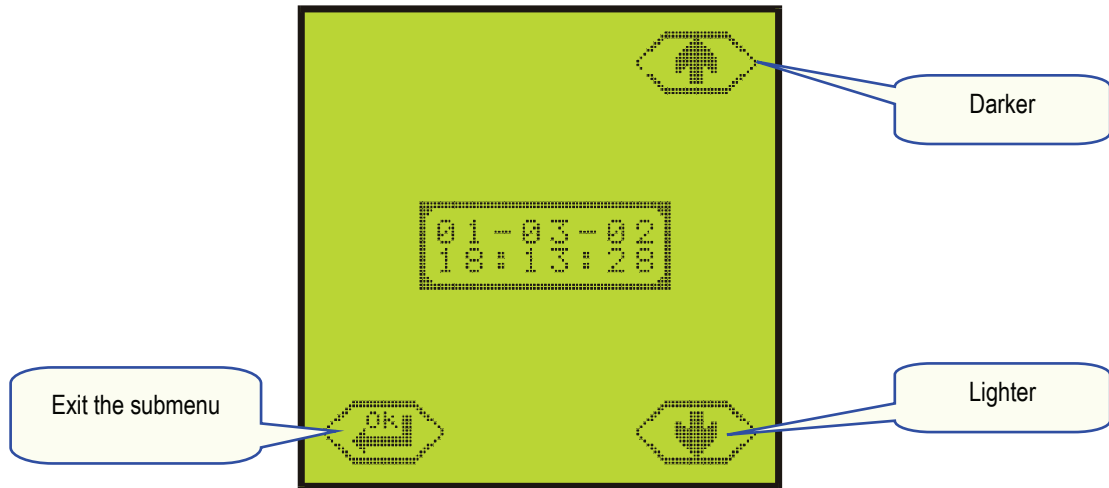NOTE: The date stamp is configured in mm:dd:yy

#### 4.5.6. PROGRAMMING THE STAND-BY MODE

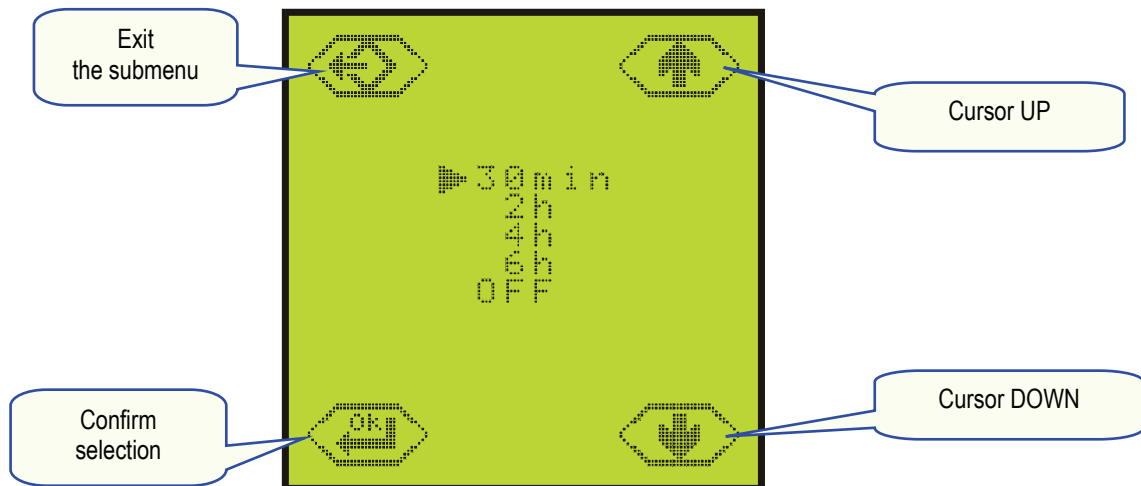
#### 4.5.7. USING THE MAINTENANCE PROGRAM SCREEN

The Maintenance Program screen shows the number of cycles remaining before replacement of the filter (400) and the door gasket (1000), as well as when general servicing (4000) is required. The three counters decrease in value after each cycle.

When one of the counters reaches 0, a corresponding message appears at the bottom of the selection screen. It is not possible to run a new cycle (the selection button will disappear) unless the reading of the message has been confirmed by pressing OK. The counter automatically resets.


If one of the three maintenance operations is completed before the respective counter reaches 0, reset the counter manually. Place the cursor in front of the operation using the icons UP and DOWN and reset it by pressing OK.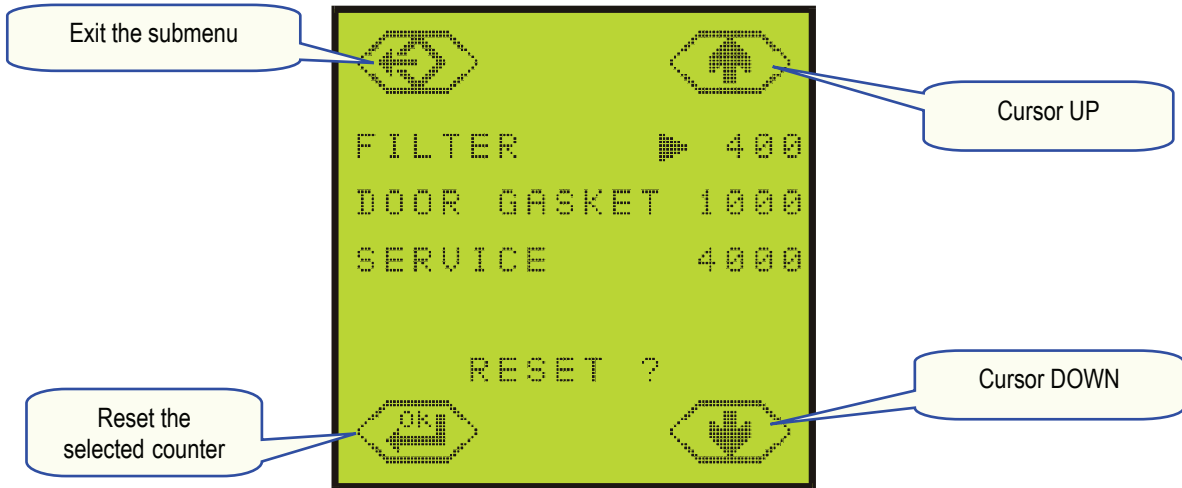

#### 4.5.8. RUNNING SERVICE PROGRAMS

Use this submenu to select and run the Bowie & Dick and vacuum test cycles.

The diagnostic function is reserved for technical service personnel and requires an access code.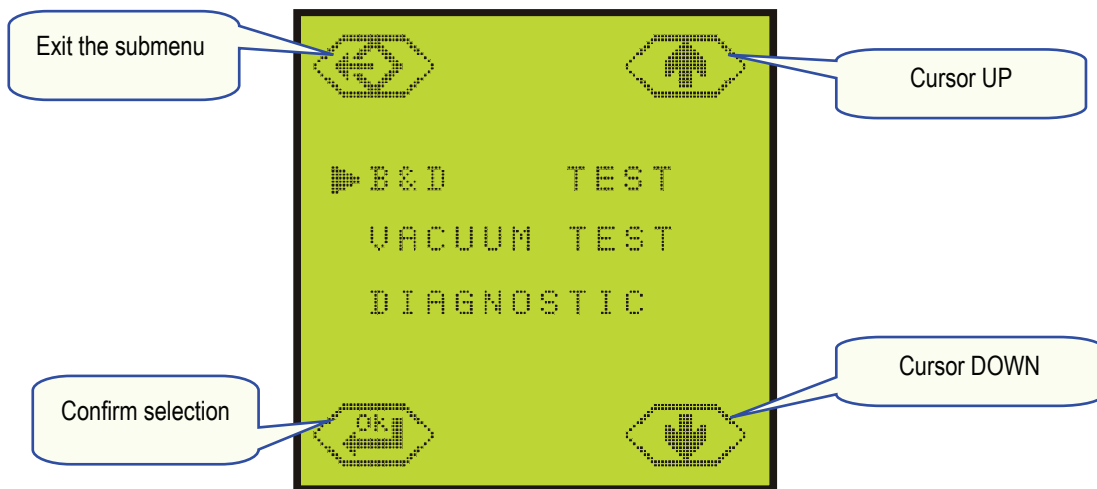

## 4.6. CYCLE SELECTION


### 4.6.1. CYCLE TABLE

CYCLES	Standard B 273°F	Extended B 273°F	Standard B 250°F Textile	B&D test	Air leak Vac. test
Temperature	273°F	273°F	250°F	273°F	--
Pressure	31.5 psi	31.5 psi	17 psi	31.5 psi	-12.3 psi
Duration of the sterilization phase	4 min.	18 min.	30 min.	3 min. 20 sec.	16 min.
Duration of the drying phase	15 min.	15 min.	20 min.	4 min.	--
Max mass of the load	13 / 4.5 lbs Metal / Textile	13 / 4.5 lbs Metal / Textile	13 / 4.5 lbs Metal / Textile	Empty	Empty
Total duration empty-full load	32 min. - 52 min.	46 min. - 66 min.	48 min. - 68 min.	26 min.	25 min.
Full solid load	<b>YES</b>	<b>YES</b>	<b>YES</b>		
Full porous load (textile)	<b>YES</b>	<b>YES</b>	<b>YES</b>		
Small porous items	<b>YES</b>	<b>YES</b>	<b>YES</b>		
Hollow A (handpieces) - B	<b>YES</b>	<b>YES</b>	<b>YES</b>		
Unwrapped	<b>YES</b>	<b>YES</b>	<b>YES</b>		
Single / double wrapped	<b>YES</b>	<b>YES</b>	<b>YES</b>		
<u>Load:</u> This sterilizer can sterilize the following type of loads: full metal, textile, hollow A-B, plastics, rubber, etc. (unwrapped, single and double wrapped). <u>NOTE:</u> Instructions provided by the respective manufacturers of items that are sterilized must be followed in each case.					

FOR LISA MB 22 ONLY

The appropriate cycle for the relevant load type and sterilization mode can be determined from the table above. Various load types can be sterilized at the same time but a universally compatible cycle must be chosen. **Do not exceed the maximum load weight.**

All sterilization cycles have the same profile. Only the duration of the sterilization plateau time, the drying time and the sterilization plateau temperature vary.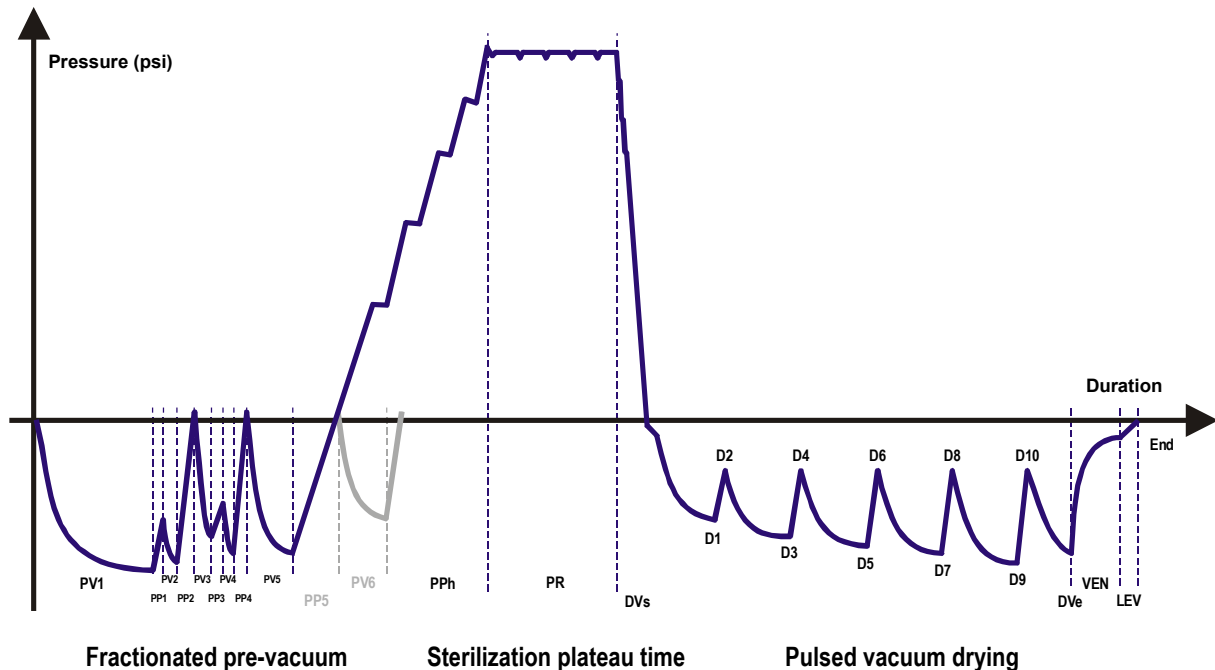

**Legend:**

PV1..6 : Vacuum pulse  
PR : Process

PP1..5 : Pressure pulse  
DV : Vacuum drying



PPh : Pressure pulse and heating  
VEN : Ventilation  
LEV : Levelling

English

### 4.6.2. STARTING A CYCLE

Exit from the stand-by mode if necessary...

Enter cycle selection menu...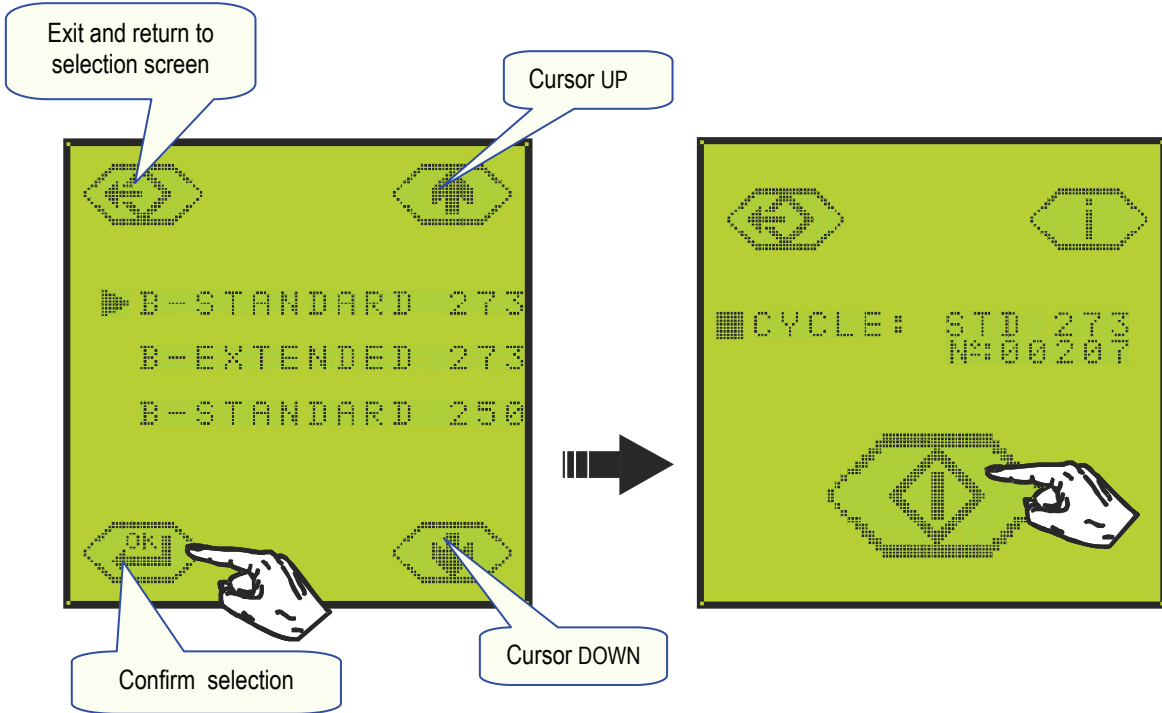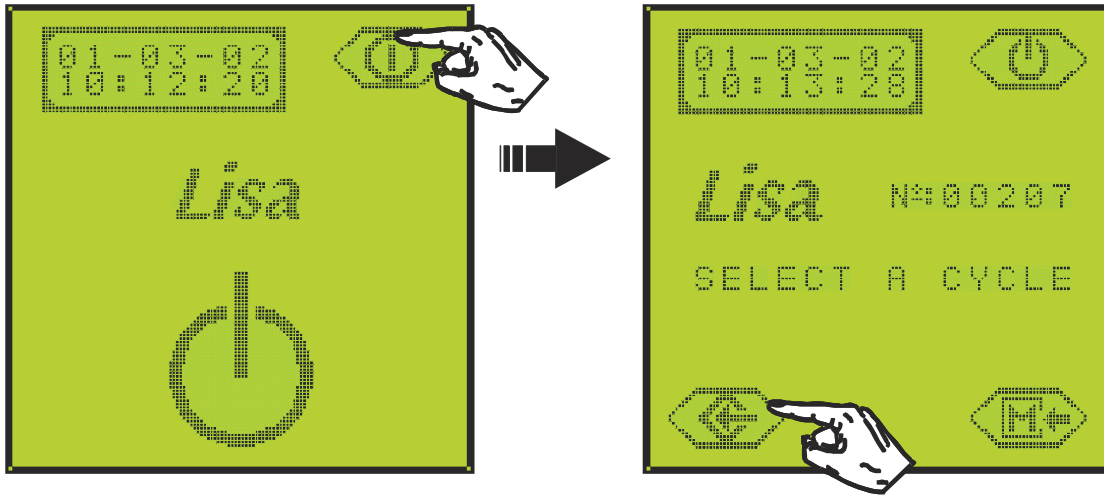

Position the cursor in front of the required cycle.  
Confirm your selection by pressing OK.

Start the selected cycle

### 4.6.3. UNDERSTANDING THE CYCLE PROCEDURE

The following information displays during the cycle:

You can access the technical parameters at any time by pressing INFO (i) (see *Locating cycle data information*). If needed, interrupt the cycle by pressing STOP for 3 seconds. An audible tone will confirm your request. This is followed by a 2-minute interval necessary to reinitialize the system and revert to atmospheric pressure in the chamber (see *Stopping manually*)

### END OF THE CYCLE

At the end of the cycle, the microprocessor analyzes the temperature, pressure, and duration of all cycle phases to confirm proper execution of the cycle.

**COMMENTS:** If the door remains locked at the end of the cycle, the vacuum pump continues to operate. Every 10 minutes the pump will run for 40 seconds in order to ventilate the chamber, preventing the build-up of moisture. This operation continues until the temperature of the chamber drops below 140°F. Before the door is opened, it is possible to access the cycle data by pressing INFO (i) (see *Locating cycle data summary*).

### 4.6.4. COPIES OF CYCLE PRINTOUTS

It is possible to get copies of the last cycle printout prior to door opening.


Press ...

...Select the quantity needed: 1 to 9, Print by pressing

#### 4.7. MANUAL STOP

If needed, a cycle can be aborted at any moment by pressing STOP for 3 seconds. An audible tone will confirm the request.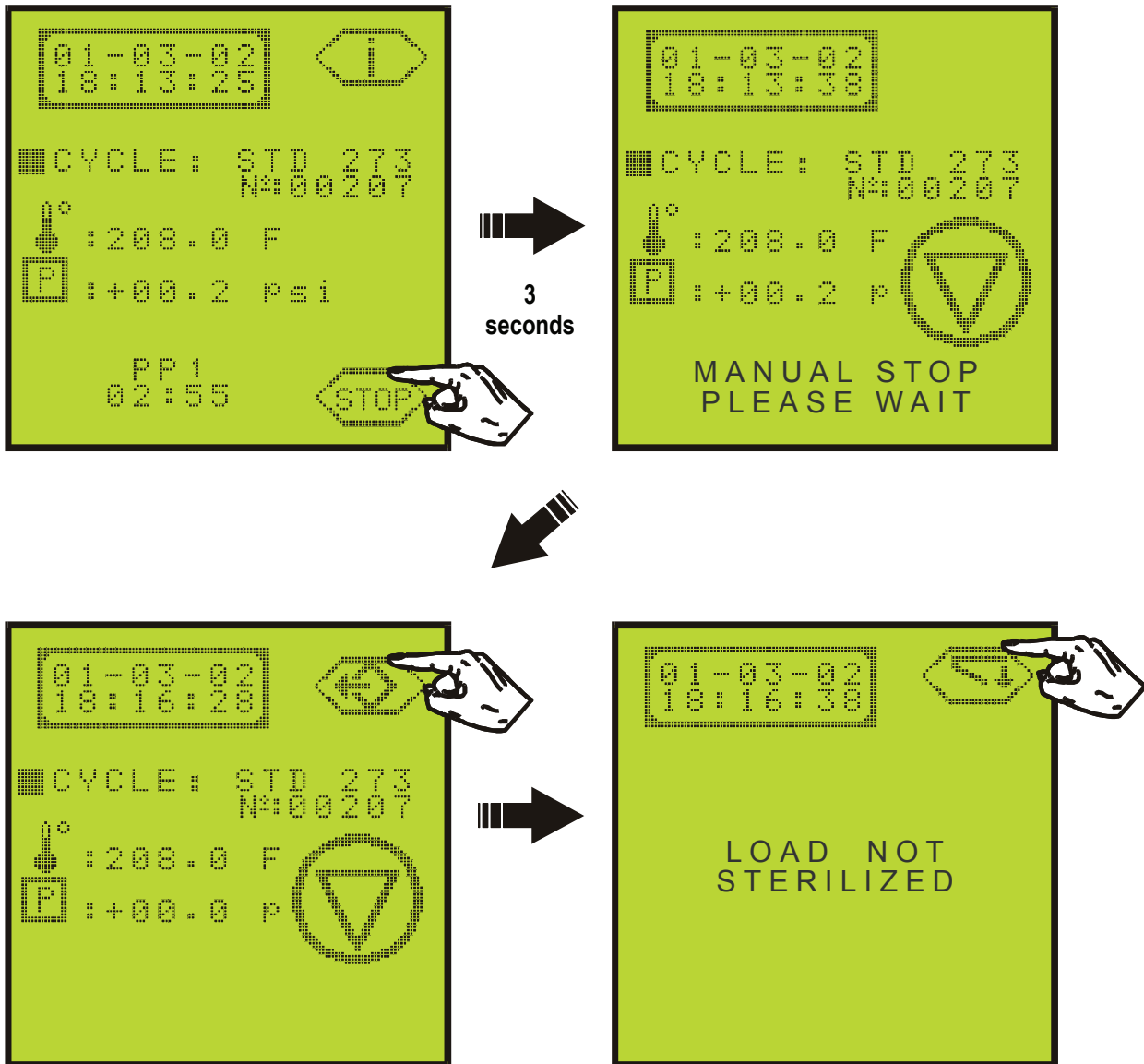


A phase of approximately 2 minutes is required to reinitialize the system and return the chamber to atmospheric pressure. The door can be opened when the door open button appears on the display at the end of this phase.

If the cycle is aborted after the PR phase (plateau period), the following message will be displayed and printed :

! PLATEAU PERIOD COMPLETED BUT !  
! LOAD NOT DRIED: IMMEDIATE USE ONLY !


### 4.8. LOCATING CYCLE DATA INFORMATION

You can access technical data at any time during the cycle by pressing INFO (i).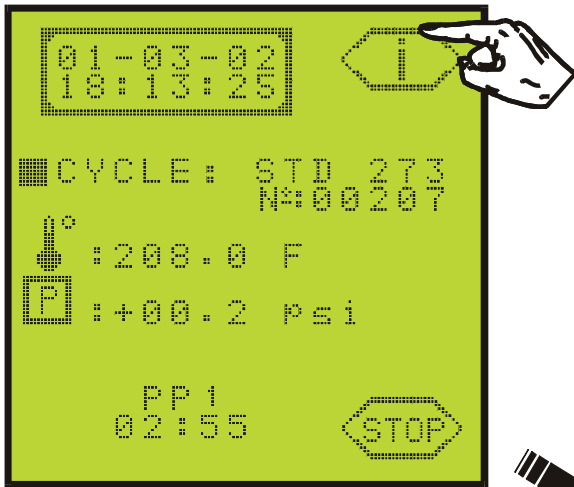

The following data displays.


- Date
- Time
- Chamber pressure
- T° calculated/Pressure
- T° of the steam
- T° chamber heater
- T° steam generator
- Phase in progress
- Duration of the phase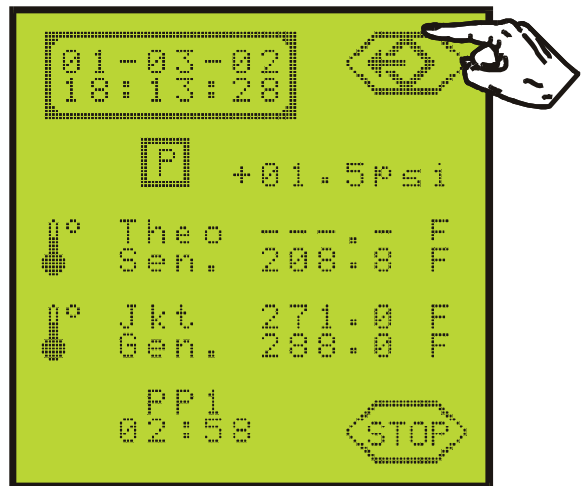

English

Theo = Theoretical  
 Sen = Sensor  
 Jkt = Jacket  
 Gen = Generator


Exit and return to the current cycle display.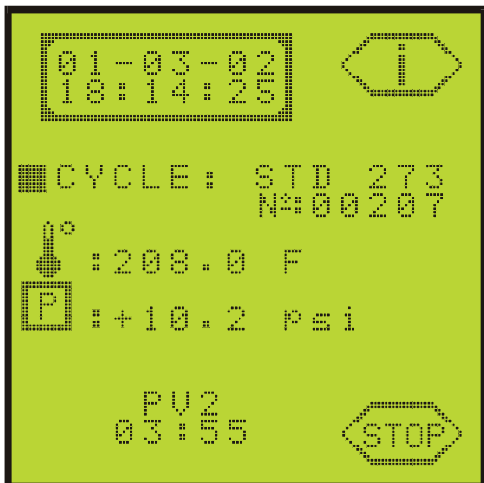


### 4.9. LOCATING CYCLE DATA SUMMARY (end of the cycle)

Before opening the door, access the technical data summary of the completed cycle by pressing INFO (i).


Advance to next screen.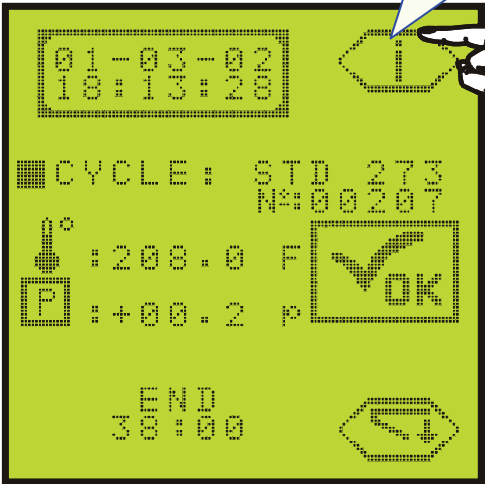

The first screen lists the pressure and duration of each phase.  
 PV= vacuum pulse  
 PP= pressure pulse  
 PPh = pressure pulse and heating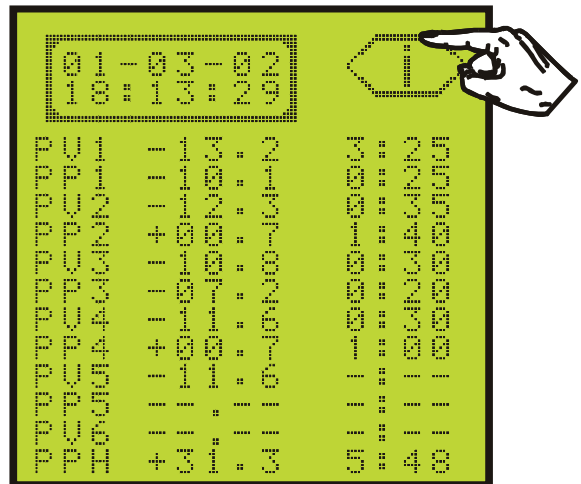

Exit and return to Cycle End screen

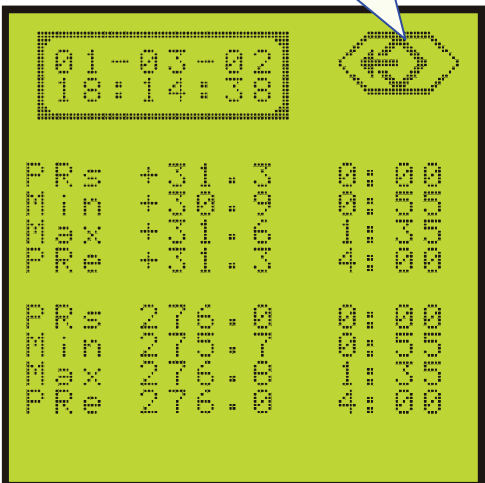The second screen indicates the pressure and temperature relating to the sterilization phase:  
 Measured pressures: begin/ min / max / end  
 Measured temperatures: begin/ min / max / end  
 PRs = Process Start  
 PRe = Process End

## 5. MESSAGES

A number of messages can appear at the beginning or at the end of the cycle. This is merely information and not an alarm. The user can continue to operate the sterilizer.

During a cycle, the microprocessor continuously analyzes some of the parameters. The cycle interrupts immediately and an alarm displays if the cycle does not execute properly.

### Message display

### Message list


Message	Description	Action
FILL THE WATER TANK	The minimum water level has been reached in the main tank.	Fill the main tank with distilled or demineralized water as described in <i>Filling the main water tank</i> .
DRAIN THE USED WATER TANK	The maximum water level has been reached in the used water tank.	Drain the tank as described in <i>Draining the used water tank</i> .
CLOSE THE DOOR	The door is not closed properly.	Close the door.
DOOR LOCKING PROBLEM	The door could not be locked properly.	Clean the door seal and check positioning. Monitor: → Call service if the message persists.
T <sub>theo</sub> /T <sub>sen</sub> DIFF IN STERIL. PHASE  T <sub>theo</sub> = theoretical temperature T <sub>sen</sub> = sensor temperature	During sterilization phase: 3.6°F or greater difference between measured sensor temperature and theoretical temperature (deducted from the pressure).	The sterilization cycle was completed successfully because at the beginning of the cycle, the microprocessor evaluates the vacuum level and stops the cycle if it is improper. Monitor: → Call service if the message persists.
Temp FLUCTUATION IN STERIL. PHASE	During sterilization phase: fluctuation of +/- 2.7°F of the measured sensor temperature and/or the theoretical temperatures (deducted from the pressure).	The sterilization cycle was completed successfully because at the beginning of the cycle, the micro-processor evaluates the vacuum level and stops the cycle if it is improper. Monitor: → Call service if the message persists.

## 6. ALARMS


The micro-processor continuously analyzes some of the cycle parameters. If an improper condition is detected, the cycle is interrupted and an alarm displays. This is followed by a phase of approximately two minutes required to reinitialize the sterilization system and return the chamber to atmospheric pressure.

**If the cycle is interrupted after the PR phase (plateau period), following message will be displayed and printed :**  
**! PLATEAU PERIOD COMPLETED BUT !**  
**! LOAD NOT DRIED: IMMEDIATE USE ONLY !**

The Alarm screen appears.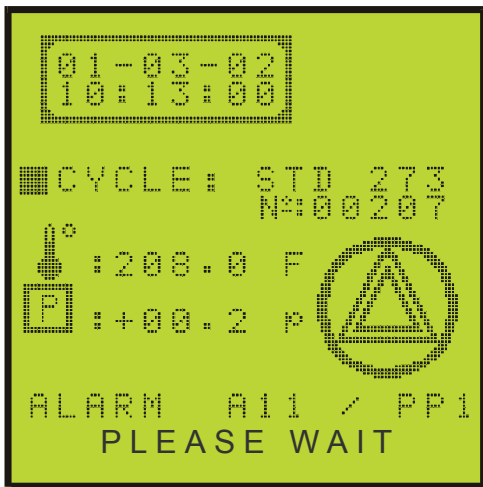



Confirm the alarm reading at the end of the re-initialization phase by pressing exit submenu button.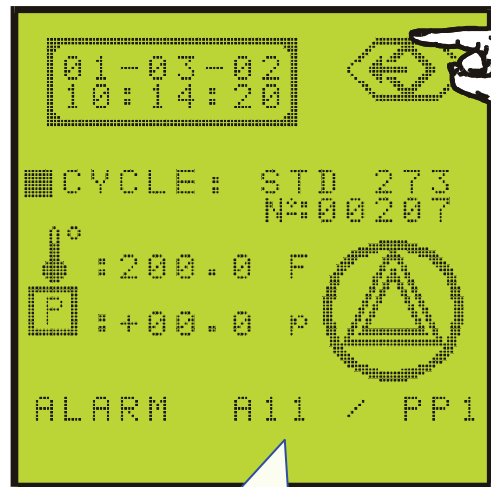

Code of the alarm / interrupted phase see *Describing the alarms*.

English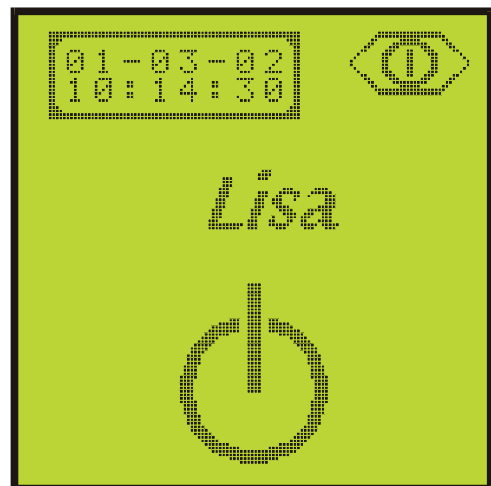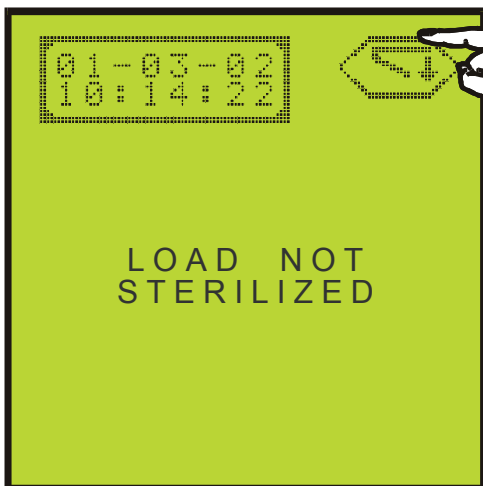

## 6.1. DESCRIPTION OF THE ALARMS

Number	Description	Action
<b>Electricity</b>		
A01	Electrical failure or significant drop in voltage occurred during the cycle.	Repeat the cycle.
<b>Sterilization chamber</b>		
A10	The time spent to reach the sterilization plateau is too long (overload, leaks, etc.).	Clean the door seal and chamber face; repeat the cycle. If the problem persists → Call service
A11	The temperature of the chamber-heating element is above the nominal value.	Repeat the cycle. If the problem persists → Call service
A12	The temperature of the chamber-heating element is below the nominal value.	Repeat the cycle. If the problem persists → Call service
A13	During the sterilization process, the pressure measured in the chamber is above the maximum threshold.	Repeat the cycle. If the problem persists → Call service
A14	During the sterilization process, the pressure measured in the chamber is below the minimum threshold.	Clean the door seal and chamber face; repeat the cycle. If the problem persists → Call service
A15	During the sterilization phase, the temperature of the steam is below the minimum threshold.	Clean the door seal and chamber face; repeat the cycle. If the problem persists → Call service
A16	During the sterilization phase, the temperature of the steam is above the maximum threshold.	Repeat the cycle. If the problem persists → Call service
A17	The temperature sensor of the chamber-heating element is broken or disconnected.	→ Call service
A18	The internal temperature sensor (steam) of the chamber is broken or disconnected.	→ Call service
<b>Steam generator</b>		
A21	The temperature of the steam generator is above the maximum threshold.	Repeat the cycle. If the problem persists → Call service
A22	The temperature of the steam generator is below the minimum threshold.	Repeat the cycle. If the problem persists → Call service
A23	The temperature sensor of the steam generator is broken or disconnected.	→ Call service

## DESCRIPTION OF THE ALARMS (continued)

Number	Description	Action
<b>Vacuum</b>		
A31	During a vacuum phase, the pressure would not drop below - 2.9 psi.	Clean the door seal and chamber face; repeat the cycle. If the problem persists → Call service
A32	During a vacuum phase, the pressure would not drop below - 7.2 psi.	Clean the door seal and chamber face; repeat the cycle. If the problem persists → Call service
A33	The pre-vacuum phase was not successful.	Clean the door seal and chamber face; repeat the cycle. If the problem persists → Call service
A34	The last 10 cycles have required a 6th additional vacuum pulse.	Clean the door seal and chamber face; the alarm is displayed only once every 10 cycles should problem persist. If the problem persists → Call service
<b>Door locking</b>		
A52	<ol style="list-style-type: none"> <li>1) The door locking system is blocked during opening and closing phases.</li> <li>2) The door-locking switch is open during the cycle.</li> </ol>	→ Call service
<b>Electro-valves</b>		
A63	Lack of steam pressure in the sterilizer chamber.	→ Call service

**CAUTION:** If any alarm occurs before the end of the sterilization phase, the load is not sterile and must be reprocessed.

## 7. USER MAINTENANCE

A distinction must be made between two levels of maintenance; that performed regularly by the user and preventive maintenance carried out by an authorized A-dec/W&H Dealer.

**NOTE:** Disconnect the power supply before examining the sterilizer.

### 7.1. PERFORMING MAINTENANCE


Frequency /	Number of cycles	Operation	Spare number	Section
Weekly	-	Cleaning the door seal.	-	§7.2
		Cleaning the chamber, the trays and the rack.	-	§7.3
		Cleaning the external surfaces.	-	§7.4
-	400	Replacing the air filter.	54.0067.00	§7.5
-	1,000	Replacing the door seal.	54.0014.00	§7.6
-	4,000	Servicing by an authorized A-dec W&H Dealer.	-	§8

The maintenance submenu indicates the number of cycles remaining before replacement of the air filter and the door seal and when general servicing is required (see Using the maintenance program screen). The three counters decrease in value after each cycle.

When one of the counters reaches 0, a corresponding message appears at the bottom of the selection screen.

It is not possible to run a new cycle (the selection button disappears) unless the reading of the message has been confirmed by pressing OK. The counter then automatically resets.

If one of the three maintenance operations is completed before the respective counter reaches 0, reset the counter manually. Place the cursor in front of the operation UP and DOWN and reset by pressing OK.

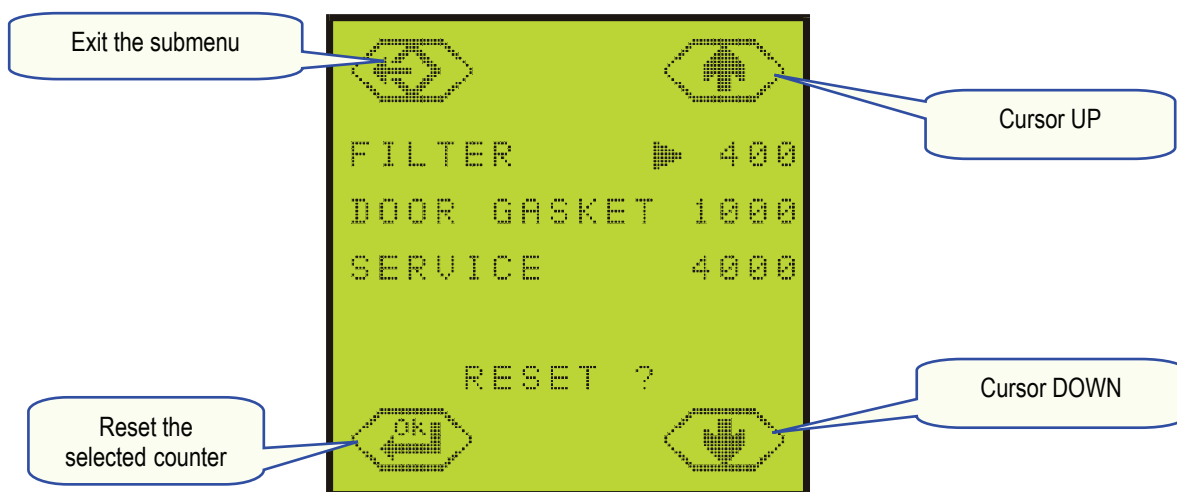## 7.2. CLEANING THE DOOR SEAL

Clean the door seal and the porthole with a lint free cloth saturated with alcohol. The porthole can also be cleaned with a non-abrasive detergent.

## 7.3. CLEANING THE CHAMBER, TRAYS, AND TRAY HOLDER

Remove the trays from the chamber.

Disconnect and remove the rack.

Clean the chamber with a damp sponge moistened with a detergent or scouring agent if necessary.

Rinse with a damp sponge to remove all traces of the cleaning agent.

Apply the same procedure for the rack, trays (cassettes).

**NOTE:** Ensure that you clean all around the sterilizer chamber. Do not bend or damage the temperature sensor at the bottom of the chamber. **Never use disinfectants to clean the chamber.**


## 7.4. CLEANING THE EXTERNAL PARTS

Clean the external parts with a damp cloth and mild detergent. Never use scouring agents or highly abrasive products.

**NOTE:** Do not use copious amounts of water to wash the sterilizer as this may damage the electrical components and safety mechanisms. Take care not to scratch the plastic film in front of the touchscreen.

## 7.5. REPLACING THE FILTER

1. Open the service door.
2. Unscrew the filter by hand (counter-clockwise).
3. Insert and manually screw the new filter into position; tighten filter using moderate force.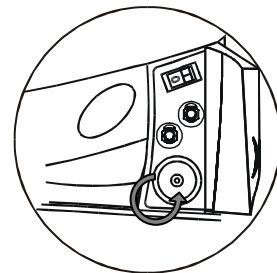


## 7.6. REPLACING THE DOOR SEAL


1. Open the door of the sterilizer.
2. Remove the door seal by hand.
3. Clean the seal and seat carefully with a cotton swab moistened with isopropyl alcohol.
4. Moisten the new seal with water.
5. Insert the seal in the sequence illustrated in the following diagram:

Insert the seal starting as follows:


Up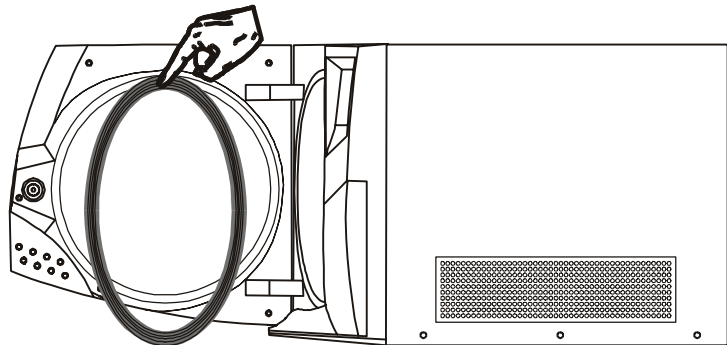

Down

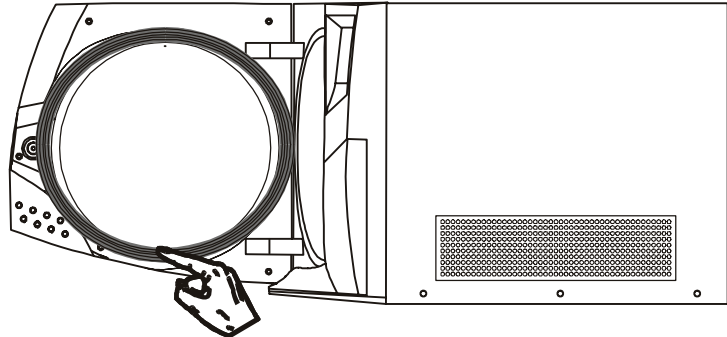Left and right

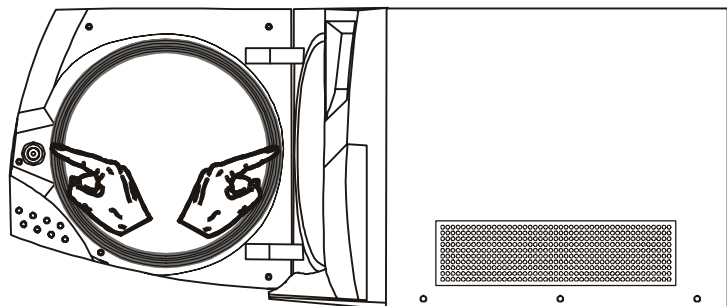---

## 8. SERVICING CONDUCTED BY THE APPROVED TECHNICIAN

Service is essential for proper operation of the sterilizer. We recommend servicing by an authorized A-dec/W&H Dealer every 4000 cycles.

### Checklist:


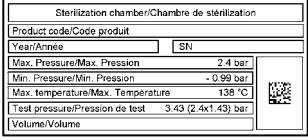
- Replace the five electric-valves
- Replace the vacuum pump membrane kit
- Replace the water filter
- Replace the door seal
- Replace the filter
- Replace the steam generator heating element
- Clean the sterilization chamber
- Clean the sterilization chamber filter
- Clean the steam generator filter
- Clean the condenser and condenser fan
- Check the pneumatic connections
- Check the electrical connections
- Check the door locking system

## 9. BREAK-DOWN GUIDE

The problem list below is limited because most defects and incidents are covered in messages and alarms.

PROBLEM	POTENTIAL CAUSES	SOLUTIONS
The sterilizer remains switched OFF.	<ul style="list-style-type: none"> <li>▪ The main switch or network circuit breaker is open (Off).</li> <li>▪ No voltage at the socket.</li> <li>▪ The power supply cable is not properly connected.</li> </ul>	<ul style="list-style-type: none"> <li>▪ Activate the main switch or network circuit breaker (ON).</li> <li>▪ Check the electrical circuit.</li> <li>▪ Plug in the cable properly.</li> </ul>
Water leaking at the front of the sterilizer.	<ul style="list-style-type: none"> <li>▪ Leaks via the door seal.</li> <li>▪ Punctured or disconnected hose.</li> </ul>	<ul style="list-style-type: none"> <li>▪ Clean the door seal (see <i>Cleaning the door seal</i>).</li> <li>▪ → Call service</li> </ul>
At the end of the cycle, water remains within the chamber and the load is not perfectly dry.	<ul style="list-style-type: none"> <li>▪ Poorly levelled machine.</li> <li>▪ Overloaded chamber.</li> <li>▪ Load incorrectly positioned.</li> </ul>	<ul style="list-style-type: none"> <li>▪ The sterilizer must be installed on a level surface.</li> <li>▪ Comply with the maximum load weight limit for each type of load (see <i>Using the cycle table</i>).</li> <li>▪ Follow the recommendations listed in <i>Appendix 2</i>.</li> </ul>
Humidity in the packaging or in the load.	<ul style="list-style-type: none"> <li>▪ Overloaded chamber.</li> <li>▪ Load incorrectly positioned.</li> </ul>	<ul style="list-style-type: none"> <li>▪ Comply with the maximum load weight limit for each type of load (see <i>Using the cycle table</i>).</li> <li>▪ Follow the recommendations listed in <i>Appendix 2</i>.</li> </ul>
Oxidation or spots on instruments.	<ul style="list-style-type: none"> <li>▪ Use of poor quality water or water containing chemical substances.</li> <li>▪ Organic or chemical residues on the instruments.</li> <li>▪ Contact between various materials.</li> <li>▪ Lime deposits in the chamber.</li> </ul>	<ul style="list-style-type: none"> <li>▪ Drain both water tanks. Use good quality distilled or demineralized water (described in <i>Appendix 6</i>).</li> <li>▪ Clean and rinse all instruments with demineralized water (<i>Appendix 2</i>). Remove all traces of disinfectants.</li> <li>▪ Interleave with tissues.</li> <li>▪ Clean the chamber.</li> </ul>
Instruments turning brown or black.	<ul style="list-style-type: none"> <li>▪ Incorrect temperature selected.</li> </ul>	<ul style="list-style-type: none"> <li>▪ Consult the table in <i>Using the cycle table</i>.</li> <li>▪ Follow the instrument manufacturer instructions.</li> </ul>

## Appendix 1 TECHNICAL CHARACTERISTICS
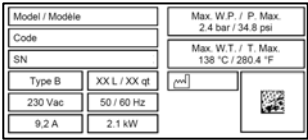
<p><b>Electrical supply</b></p> <p><b>Sterilizer:</b></p> <p>Working temperature / Humidity Storage temperature / Humidity Min. atmospheric pressure Nominal voltage : Max. absorbed power : Max. current : Dimensions overall : Max. space required : Clutter of the door movement : Weight empty : Max. weight in working condition, fully loaded : Max. heat output : Max. noise level :</p>	<p>Single phase 230/240 VAC <math>\pm</math> 10% -50/60Hz – 2100W</p> <p><b>Name plate on the back side:</b></p> <p>50°- 104°F / 0-90%. -4°- 140°F / 0-90% (empty) 7.3 PSI 230/240 VAC 2100 W 10.3 A W : 18 in / H : 16 in / D : 24 ½ in W : 19 in / H : 18 in / D : 26 ½ in W : 14 in/ H : 16 in / D : 14 in 125 lbs 155 lbs 2846 BTU/hr &lt; 50dB</p> 
<p><b>Steam generator</b></p> <p>Power / Voltage : Max. pressure / Max. temperature : Pressure safety valve :</p>	<p><b>Name plate on the steam generator:</b></p> <p>1700 W 58 PSI/ 302°F 73 PSI</p> 
<p><b>Sterilization chamber</b></p> <p>Power / Voltage : Max. pressure / Max. temperature : Pressure safety valve : Total volume : Usable space (identical for all cycles) : Filter :</p>	<p><b>Name plate on the chamber:</b></p> <p>1000 W 35 PSI / 280°F 35 PSI 23 qts Dia.: 9.8 in / Depth: 17 ½ in 17 qts W: 7.7 in / H: 8.1 in / D:15 ¾ in 0.3 µm</p> 
<p><b>Distilled water (or demineralized) :</b></p> <p>Water quality : Min. / Max. consumption : Double tank / working range :</p>	<p>See <i>Appendix 5</i> 0.2 qt / 0.4 qt (full porous load) Min 8 cycles (full porous load)</p>
<p><b>Connections</b></p>	<p>Parallel printer port</p>
<p><b>Miscellaneous</b></p>	<p>Fully micro-processor driven and controlled / touchscreen Programmable stand-by mode</p>

FOR LISA MB 22 ONLY

English

## Appendix 2      PREPARATION OF THE LOAD

### 1.      **Cleaning**

Thorough cleaning of all items prior to sterilization is imperative to ensure effective sterilization. If visible debris and bio-burden is not removed through cleaning prior to sterilization, it will interfere with microbial inactivation and compromise the sterilization process!

- If possible clean instruments immediately after use; always follow instrument manufacturer instructions.
- Lubricate dental handpieces **after** cleaning, **prior to** sterilization, in accordance with the handpiece manufacturer's instructions.

### 2.      **Use of rust inhibitors**

If rust inhibitors are applied to items prior to sterilization (e.g., surgical milk dips) follow the manufacturer's instructions for the product.

- Thoroughly drain excess rust inhibitor from the load and allow product to dry prior to placement in the sterilizer.
- Always wrap or pouch items that have been treated with rust inhibitors to limit exposure to internal sterilizer components.

### 3.      **Loading the sterilizer chamber**

- Do not overload the chamber; adhere to the maximum load weight limits of 10 lbs metal / 3 lbs textile.
- Always use the chamber rack to allow adequate steam circulation.
- **Place pouched items on trays with the paper side facing up.**
- Do not overload trays; spread single items on multiple trays.
- Place items in such a way so as to allow steam to circulate properly.
- Place cassettes in the vertical position (if possible) to enhance drying.
- Place empty containers or non-perforated trays upside down to prevent accumulation of water.
- Items made from different materials (stainless steel, carbon steel, etc.) must be placed on separate trays.
- If instruments are manufactured from carbon steel, paper should be placed between them and the sterilizer tray.
- Sterilize hinged instruments in the opened position (e.g., forceps).
- Wrap items with porous wrapping materials to facilitate steam penetration and drying (e.g., sterilization bags for autoclaves).

#### Tubes:

- ◆ Rinse, drain and dry after washing.
- ◆ Place tubes on a tray allowing the ends to remain open; do not bend.

#### Packs:

- ◆ Place packs vertically (if possible) and ensure adequate spacing in-between packs.  
**Ensure packs do not touch the sterilizer chamber walls.**

#### Wrapped items:

- ◆ Sterilization bags should be placed on trays allowing adequate space in-between bags.  
**Place bags with the paper side facing up.**

## Appendix 3 STERILIZATION MONITORING

- **Biological indicators**

Biological indicators (BIs) (i.e., spore tests) are the most accepted method for monitoring the sterilization process because they assess it directly by killing known highly resistant microorganisms. Place biological indicators inside of wrapped or pouched loads and inside of containers such as cassettes on the lower tray or rack position as close as possible to the sterilizer chamber door. Follow local/national requirements on the frequency and procedure of testing. In case a biological indicator shows a positive result (not a sufficient number of microorganisms have been killed during the sterilization process), consult local/national guidelines that have been developed to follow in the event of a positive test.

- **Chemical indicators**


Chemical indicators, internal and external, use sensitive chemicals to assess physical conditions (e.g., time and temperature) during the sterilization process. Place chemical indicators on (external) or inside of (internal) wrapped or pouched loads and containers such as cassettes. If only one set of indicators is used for a specific load, place the item carrying the indicator(s) on the lower tray or rack position as close as possible to the sterilizer chamber door. Follow local/national requirements on the frequency of testing. In case chemical indicators indicate inadequate processing, items in the load should not be used until reprocessing.

- **Bowie & Dick Test**

The Bowie & Dick Test (B&D) is made of several sheets of paper and/or foam wrapped in a small pack in the middle of which there is a chemical heat-sensitive indicator sheet. Bowie & Dick type test packs detect air leaks, inadequate air removal and steam penetration for vacuum-assisted steam sterilizers.

Performing the test:

1. Place the Bowie & Dick Test (complete pack) on the lower tray or rack position.
2. Access the "SERVICE" sub-menu, select and initiate the B&D test cycle.

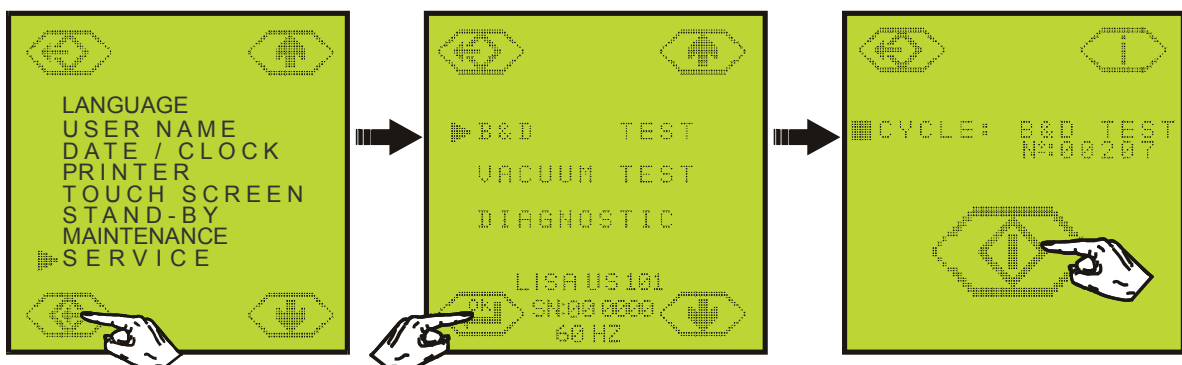3. Once the cycle is completed, open the sterilizer door and remove the test pack; caution: the pack will be very hot!
4. Remove the indicator sheet from the center of the pack and check the change in color as explained below:

**Test passed:**

The entire surface of the indicator sheet has changed color.

**Test failed:**

Certain areas of the indicator sheet have not changed color.

Any unexpected color change, such as the center of the test sheet being paler or a different color than the edges indicates that there was an air pocket present during the cycle due to sterilizer malfunction. Follow local/national requirements on the frequency of testing.

## Appendix 4 VACUUM TEST


This test is used to validate the sterilizer in terms of vacuum pump performance and tightness of the pneumatic circuit.

The pressure profile of the vacuum test cycle is as follows:

Vacuum phase to P1 = - 12.3 psi


Stabilization period of 5 min. => T2; reading of P2

Testing period of 16 min. => T3; reading of P3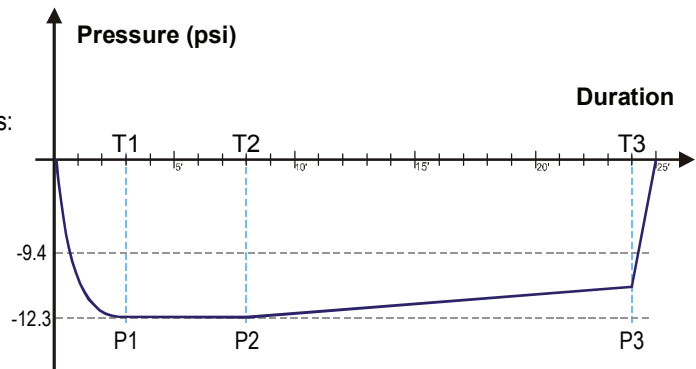

### Performing the test

- Select and start the Vacuum Test cycle from the Service submenu.

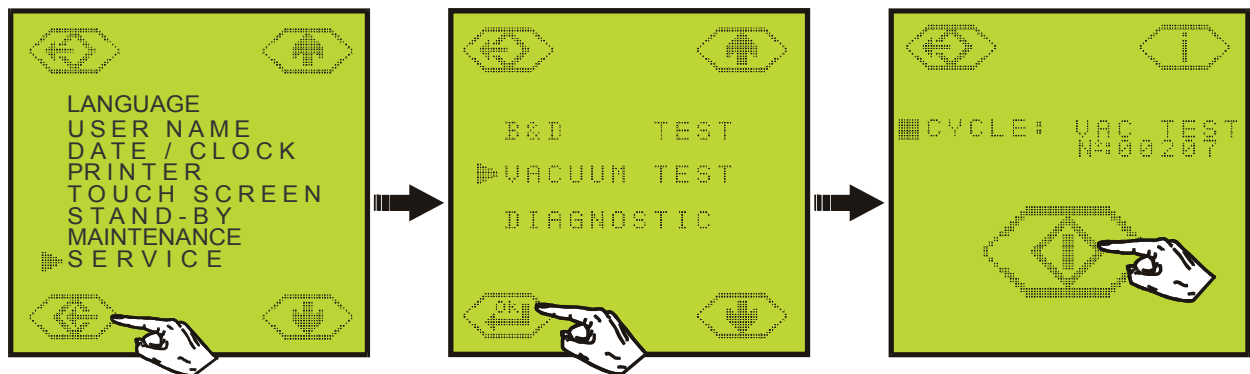English

- The microprocessor makes the following calculation:  $P3 - P2$ . The result must be less than 0.29 psi \*.
- Test results are displayed at the end of the test cycle.

**\*NOTE:** According to the EN13060 standard, an independent external 1 mbar (0.001bar) accurate pressure sensor shall be connected in order to measure  $P3-P2$  lower than 13 mbar (0.013bar). The sensor fitted to the LISA sterilizer is 10 mbar (0.01 bar) accurate and therefore with the above comments considered  $P3-P2 < 20$  mbar (0.02bar = 0.29psi).

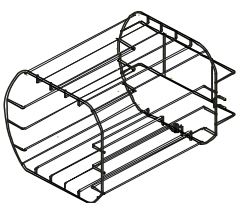
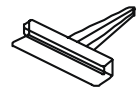
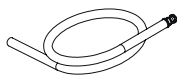
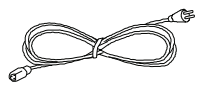
## Appendix 5 WATER QUALITY

The table below lists the quality of water to be used for steam sterilization as taken from the European standard EN13060.

	<b>Feed Water</b>
Evaporate residue	≤ 10 mg/l
Silicon oxide, SiO <sub>2</sub>	≤ 1 mg/l
Iron	≤ 0.2 mg/l
Cadmium	≤ 0.005 mg/l
Lead	≤ 0.05 mg/l
Other heavy metals, excluding iron, cadmium, lead	≤ 0.1 mg/l
Chloride	≤ 2 mg/l
Phosphate	≤ 0.5 mg/l
Conductivity (at 20°C)	≤ 15 µs/cm
pH value	5 to 7
Appearance	Colorless, clean, without sediment
Hardness	≤ 0.02 mmol/l

## Appendix 6 ACCESSORIES

DESCRIPTION		SPARE NUMBER
Aluminum tray		54.0019.00
Reversible rack		54.0018.00
Cassette holder		54.0016.00
Tray holder		54.0017.00
Drain tube		54.0029.00
Power cord		54.0046.00
Funnel		54.0024.00
Filter		54.0067.00
Door seal		54.0014.00
Lisa Log64		54.0353.00


## Appendix 7    SYMBOLS


This symbol is visible on the front of the device under the door handle.  
It recommends attention be paid to the high temperatures associated with the chamber, the door and the area around the door handle.


Attention, consult accompanying documents.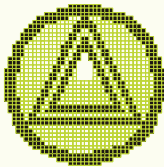


Manual Stop, Please wait

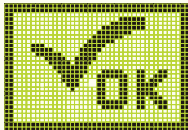Cycle properly executed

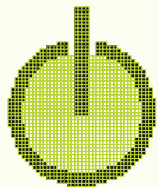Stand-by mode

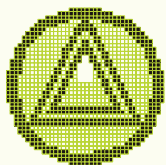Alarm; see “Description of the alarms”

## Appendix 8      WARRANTY

### A-dec|W&H Warranty

Highly qualified specialists have manufactured this A-dec|W&H product with extreme care. A wide variety of tests and inspections guarantee fault-free operation.

A-dec|W&H warrants this product against defects in material or workmanship for one year from the time of purchase. A-dec's sole obligation under warranty is to provide parts for the repair, or at its option, to provide replacement product.

The warranty does not cover damage resulting from improper installation, maintenance, accident, or misuse.

Failure to follow the instructions in the *Owner's Guide* may void the warranty.

We accept no responsibility for damage caused by improper handling or inadequate packaging during return of goods.

LISA MB 22 SN 00-0XXX

1 Year Warranty or  
1,000 cycles  
(see note)

Purchase date:

Supply depot:

Practice/Clinic/Laboratory  
address:

FOR LISA MB 22 ONLY

English

**NOTE:    Extend Your Warranty**

This warranty will be extended to 2 Years or 2,000 cycles if the A-dec|W&H LISA Installation Checklist is fully completed and a copy is either mailed or faxed to the address found on the Installation Checklist.

## Appendix 9 A-dec|W&H Lisa Installation Checklist

- |  |     |    |
|--|-----|----|
| 1. Is the countertop allocated for the LISA Sterilizer level and flat?   | Yes | No |
| 2. Does the area allocated for the LISA Sterilizer provide the recommended ventilation space?  | Yes | No |
| 3. Did the packing for the LISA Sterilizer appear undamaged?   | Yes | No |
| 4. When unpacked, did the LISA Sterilizer appear undamaged?  | Yes | No |
| 5. Did all of the accessories arrive with the Sterilizer?  | Yes | No |
| 6. Did you have all of your staff present during the in-service?   | Yes | No |
| 7. Were all sections of the owner's guide covered during the in-service?   | Yes | No |
| 8. Was a B-Standard 134 cycle run, following the instruction manual, with the tray rack and trays inserted? This cycle details each step required for setup of the sterilizer. | Yes | No |
| 9. Do you know what quality water is required for use in the sterilizer?   | Yes | No |
| 10. Have you been shown the procedure for filling and draining the main and used water tanks?  | Yes | No |
| 11. Have you been shown how to program your new LISA Sterilizer?   | Yes | No |
| 12. Have you been shown the cycle options?   | Yes | No |
| 13. Have you been shown what the Messages and Alarms mean?   | Yes | No |
| 14. Have you been shown how to manually abort a cycle?   | Yes | No |
| 15. Have you been advised of the weekly maintenance procedures?  | Yes | No |
| 16. Have you been shown how to use all of the accessories?   | Yes | No |
| 17. Have you been shown the advantages of having a data logger for your new LISA Sterilizer?   | Yes | No |

FOR LISA MB 22 ONLY

English

Lisa MB Serial Number: \_\_\_\_\_

Date: \_\_\_\_\_ Purchased From: \_\_\_\_\_ Installed By: \_\_\_\_\_

Dr./Clinic Name: \_\_\_\_\_

Address: \_\_\_\_\_

Phone: \_\_\_\_\_

Receiver's Signature: \_\_\_\_\_ Installer's Signature: \_\_\_\_\_

Upon completion of the Installation Checklist please fax a copy to:

**A-dec|W&H**  
**Fax: 503/538-0276**

-OR-

Make a copy of the Installation Checklist and mail it to:

**A-dec|W&H**  
**LISA Sterilizer Warranty**  
**2601 Crestview Drive**  
**Newberg, Oregon, 97132**

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## *Lisa* MB 17

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© 2007 BY A-DEC INC. ALL RIGHTS RESERVED.

2601 Crestview Drive, Newberg, OR 97132. Printed in the USA.

A-dec makes no warranty of any kind with regard to the content of this document, including but not limited to, the implied warranties of merchantability and fitness for a particular purpose. A-dec Inc. shall not be held liable for any errors contained herein or any consequential or other damages concerning the furnishing, performance or use of this material. The information in this document is subject to change without notice. If you find any problems with this document, please report them to us in writing. A-dec Inc. does not warrant that this document is error-free.

All other non-A-dec products or services mentioned in this document are covered by the trademarks, service marks, or product names designated by the companies marketing those products.

A word about this Owner's Guide: Every effort has been made to make this document as comprehensive and factual as possible.

We reserve the right, however, to make changes at any time without notice in prices, colors, materials, equipment specifications, models and availability. Since some information may have been updated since the time of printing, please check with A-dec for complete details.

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## Annexe 8 GARANTIE

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<h3 style="margin: 0;">A-dec/W&amp;H Warranty</h3> <p>Highly qualified specialists have manufactured this A-dec/W&amp;H product with extreme care. A wide variety of tests and inspections guarantee fault-free operation.</p> <p>A-dec/W&amp;H warrants this product against defects in material or workmanship for one year from the time of purchase. A-dec's sole obligation under warranty is to provide parts for the repair, or at its option, to provide replacement product.</p> <p>The warranty does not cover damage resulting from improper installation, maintenance, accident, or misuse.</p> <p>Failure to follow the instructions in the <i>Owner's Guide</i> may void the warranty.</p> <p>We accept no responsibility for damage caused by improper handling or inadequate packaging during return of goods.</p>	<div style="border: 1px solid black; padding: 5px; margin-bottom: 5px;">LISA MB 22 SN 00-0XXX</div> <div style="border: 1px solid black; padding: 5px; margin-bottom: 5px;"> <h3 style="margin: 0;">1 Year Warranty or 1,000 cycles</h3> <p style="margin: 0;">(see note)</p> </div> <div style="border: 1px solid black; padding: 5px; margin-bottom: 5px;">Purchase date:</div> <div style="border: 1px solid black; padding: 5px; margin-bottom: 5px;">Supply depot:</div> <div style="border: 1px solid black; padding: 5px;">Practice/Clinic/Laboratory address:</div>
--	---

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